GB English – Instructions ARX® Soft Strap

Usage and function · The ARX® Soft Strap has been developed to be used to provide a light joint support. It has a unique function where the two soft surfaces will attach onto each other without the use of a conventional hook. The material can be cut in any way. When you cut the material lint can be released from the cutting area which can attach onto the surface. This can be removed by using an adhesive tape or a textile lint roller.

The ARX® Soft Strap provides light support and heat to the body part. It's available in 30 mm, 50 mm and 110 mm width.

Material content · The strap is made from 100% polyamide micro fibers. The strap is 100% latex free and has an Oeko-Tex 100 certification.

Caring instructions and precautions · We recommend to machine wash the strap once a week in 40 degrees, using a laundry bag. Patients with sensitive skin, diabetes or poor circulation should consult a doctor before use. Patients with allergies should check the material content before use.

For more information consult www.mediroyal.se.

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