English – instructions

Knee Supports NRX · ARX · CRX – Hinged

General information - Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the textile label.

Indications - The product has been designed to provide support and stability to knee instabilities like ligament tears, meniscus injuries, hyper-extension problems, osteoarthritis and general knee instabilities that requires a higher degree of mechanical support.

Contraindications - Patients with severe cases of valgus/varus knee or severe cases of genu recurvatum usually would require a more stable solution. However this brace might be used as a complement brace to a more mechanically stable alternative.

Pre Application - To select the right size, take the circumference measurement around the knee at the height of the knee cap. Use the measurement chart to determine the most suitable size. Open all Velcro straps of the brace before application.

Adjustment - If the brace is fitted with flexible spiral stays, no adjustment is needed.

If the brace is fitted with hinges, these should be contoured to the patient anatomy to avoid pressure points. Remove the hinges from the hinge pockets. Depending on what hinge you have in your brace, study the separate "Hinge Instruction" to set range of motion.

Application - Apply the brace by either pulling it up on the leg or wrap it around. Make sure that the knee cap is placed in the center hole in the front. Start closing any straps that are on the back side of the brace. Make sure not to over-tighten. Continue with the straps on the lower leg and then the thigh. If the brace is fitted with a crisis-cross in the knee fold, continue to point 7 "Adjustment of the Hyper-X strap". Once all straps have been closed, move around with the brace on a few minutes. If necessary adjust the straps.

Adjustment of the Hyper-X strap - If your brace has a crisis-cross strap in the knee fold it can be used to control hyper-extension of the knee. This is quite common when you have an anterior cruciate ligament injury. Start with applying the lower strap in the D-ring. Don't tighten, just position the strap and close it. The upper strap is connected to the lower strap and can be used to control the extension of the knee. Start by applying a little bit of tension and close the strap. Test to extend the knee. If more control is needed tighten the strap a little more. Once you have reached a control that feels ok try to walk with the brace. You should be able to walk normal and extend the knee without hyperextending. If you need to adjust, use the upper strap.

Removal - Open the Velcro straps and remove the brace. Close all straps when the brace is not in use.

Material content - Check the backside of the white label for material content.

Caring instructions and precautions - We recommend that the brace should be washed once a week. Follow the washing instructions on the white label on the product, close all straps, remove the hinges and use a laundry bag. If the brace has spiral stays they don't have to be removed.

Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on the backside of the label on the cover.

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Mediroyal – Functional Swedish Design

Product development is a combination of ideas, material, design and know-how. At Mediroyal we have more than 20 years of experience with developing products that provides both the patient and the therapist with enhanced functionality, every day. Swedish design is unique in many ways. Quality, performance, functionality and design are all features that make a difference. That's why we are proud to label our products with "Functional Swedish Design". Because orthoses is our passion.

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© Copyright Mediroyal Nordic AB, Sweden · 2017 · 25288DR · www.vajzer.se
Mediroyal Nordic AB · Box 7052 · SE-192 07 Sollentuna · Sweden