English – instructions

Mediroyal Proxi® Stable UCL Thumb

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Adjustment Proxi Stable UCL - The Proxi UCL thumb has an internal thermoplastic reinforcement that has been laminated with the external reinforcement film. The shape is pre-moulded and can be fitted immediately. If the size is too big, simply fold the thumb part and press it together carefully. That changes the thermoplastic and makes the size smaller. If you want to make the size bigger simply open it up and hold for a few seconds. The outer design can be trimmed with a pair of scissors if needed. Avoid cutting in the area of the thermoplastic and the hook and loop closures.

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Greek – Instructions

Mediroyal Proxi® Stable UCL Thumb

General information - Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopaedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the backside of the white text label.

Indications - The product has been design to provide support to the thumbs and the UCL-ligament after surgery or trauma.

Contra-indications - Patients with instable fractures should not use this brace.

Pre Application - To select the right size, take the circumferential measurement around the wrist. Use the measurement chart to determine the most suitable size. Note that the product made in left or right versions.

Adjustment Proxi Stable ULC - The Proxi UCL thumb has an internal thermoplastic reinforcement that has been laminated with the external reinforcement film. The shape is pre-moulded and can be fitted immediately. If the size is too big, simply fold the thumb part and press it together carefully. That changes the thermoplastic and makes the size smaller. If you want to make the size bigger simply open it up and hold for a few seconds. The outer design can be trimmed with a pair of scissors if needed. Avoid cutting in the area of the thermoplastic and the hook and loop closures.

An additional support there is a thumb strap that will help to immobilize the IP-joint. The strap is applied around the thumb to provide more stability and the strap is attached on top of the thumb closure strap. The strap can be used initially and when the patient needs to start flexion training of the IP the strap can be removed completely by cutting it with a pair of scissors close to the insertion.

The thumb piece has two plastic shields placed on each side of the thumb to provide stability. In between, on the volar side, there is a flexible area that can be trimmed gradually with a pair of scissors to allow more flexion.

Application - Close the thumb grip of the brace before application. Apply the thumb through the opening. Apply the strap around the wrist and re-adjust the thumb strap if needed.

Removal - Open the Velcro straps and remove the brace. Close all straps when the brace is not in use.

Material content - Check the material content on the white label.

Caring instructions and precautions - We recommend that the brace should be washed once a week. Follow the washing instructions on the white label on the product, close all straps and use a laundry bag. Never wash in temperature over 40 degrees Celsius.

Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on the backside of the label on the cover.

For more information consult www.mediroyal.se

© Copyright MediRoyal Nordic AB, Sweden - 2020 - 257340R - www.vajer.se

mediroyal

mediroyal

Mediroyal – Functional Swedish Design

Product development is a combination of ideas, material, design and know-how. At Mediroyal we have more than 20 years of experience with developing products that provides both the patient and the therapist with enhanced functionality, every day. Swedish design is unique in many ways. Quality, performance, functionality and design are all features that make a difference. That's why we are proud to label our products with "Functional Swedish Design". Because orthoses is our passion.

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CE MediRoyal Nordic AB - Box 7052 - SE-192 07 Solna - Sweden