NRX420 Patella Tendon

NRX421  
Patella Tendon Long

## GB English – instructions

### Knee Supports – Patella Tendon

**General information** - Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the textile label.

**Indications** - The product has been design to support and unload the patella tendon during knee problems like Os Schatters, Jumpers knee or Patella tendinitis.

**Contraindications** - This brace doesn't provide any support lateral or medial. If your patella problem requires medial or lateral support you should use an alternative brace. Visit www.mediroyal.se/en

**Pre Application** - To select the right size, take the circumference measurement around the knee at the height of the knee cap. Use the measurement chart to determine the most suitable size. Open all Velcro straps of the brace before application.

**Application** - The brace has a small pad that should be placed over the patella tendon. Since there are different designs application could be done differently.

**Pull-Up design**: If the brace is pulled up over the knee, position the pad just under the knee cap. Close all the straps on the brace and use the pulling straps to apply pressure just under the knee cap. Make sure that the pressure is comfortable and doesn't restrict your activity.

**Wrap Around design**: Open the brace and apply the thigh strap around the thigh. Position the pad just under the knee cap. Pull the open strap through the hole on the material and through the D-ring. Attach on the front side. Close the brace and then adjust the pressure with the straps. Make sure that the pressure is comfortable and doesn't restrict your activity.

**Removal** - Open the Velcro straps and remove the brace. Close all straps when the brace is not in use.

**Material content** - Check label for material content.

**Caring instructions and precautions** - We recommend that the brace should be washed once a week. Follow the washing instructions on the white label on the product, close all straps and use a laundry bag.

**Patients with sensitive skin, diabetes or poor circulation** should be observed. Patients with allergies should check the material content on the label.

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