General information

Read the instruction carefully. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the white textile label.

Indications

The product has been designed to treat plantar fasciitis during the night by stretching the plantar fascia. It can also be used for other indications when plantar flexion needs to be restricted.

Contraindications

Instable fractures or spasticity in the foot or ankle should not be treated with this splint.

Pre Application

To select the right size use the measurement chart.

Application

Open the straps with the plastic buckles. Adjust the length of the straps through the buckle and then fixate the padding. Position the foot in the splint. Use the straps on the side to adjust the angle of dorsiflexion that is needed. The night treatment for plantar fasciitis can take a longer time, up to three months.

Removal

Open the straps with the buckles. There is no need to open the straps on the side that regulates the dorsiflexion.

Caring instructions and precautions

The lining of the splint should be washed once a week or when needed. We recommend using mild soap and water to wipe it off and let air dry. There are replacement linings available for purchase.

The splint should not be used as an ambulatory walker. It's not designed for any weight bearing except for a few steps. Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on the label on the product.

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© Copyright MediRoyal Nordic AB, Sweden - 2016 - 25097DR - www.vajer.se