GB English – instructions

MediRoyal Back Supports

General information · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the backside of the white textile label on the cover for material content.

Indications · The product has been design to provide lumbar support and stability.

Contraindications · Patients with instable fractures should use a more stable brace.

Pre Application · To select the right size, take the circumferential measurement around the waist. Use the measurement chart to determine the most suitable size.

Adjustment · The brace are fitted with two metal splints. They are located in the back, the two open pockets. Remove the metal splints from the pocket and contour them to the lumbar area of the patient. Insert the splints again and make sure to mark what's up and down on the splint.

Application · Open the Velcro closure of the brace. Apply the main brace over the lumbar region and close in the front. Then grip the reinforcement strap and apply it in the front of the brace by the Velcro closure. Adjust compression to a comfortable level.

Removal · Open the Velcro straps and remove the brace.

Caring instructions and precautions · We recommend that the brace should be washed once a week. Follow the washing instructions on the white label on the product, close all straps and use a laundry bag.

Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on the backside of the label on the cover.

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