General information · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the textile label.

Indications · Lateral or medial epicondylitis.

Pre Application · To select the right size, take circumference measurement 5 cm below the elbow joint. Use the measurement chart to determine the right size.

Application · The pad in the brace should be placed over the highest position of the muscle. When you make a fist with your hand the pressure from the pad should be felt over the muscle. The brace is only effective during loading of the hand/arm. It shouldn't be used when you rest or sleep. Epicondylitis are usually caused by over extension or wrong position of the wrist. Therefore it might be a good idea to combine the brace with a wrist brace as well. This unloads the tendons and decreases the pain.

Caring instructions and precautions · We recommend that the brace should be washed once a week. Follow the washing instructions on the label and always use the laundry bag. Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on the label.

For more information visit www.mediroyal.se

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CRX Epi Elbow

MR2810

Mediroyal – Functional Swedish Design

Product development is a combination of ideas, material, design and know-how. At Mediroyal we have more than 20 years of experience with developing products that provides both the patient and the therapist with enhanced functionality, every day. Swedish design is unique in many ways. Quality, performance, functionality and design are all features that make a difference. That's why we are proud to label our products with "Functional Swedish Design". Because orthoses are our passion.

MediRoyal Nordic AB
Box 7052 · SE-192 07 Sollentuna · Sweden
+46 8 506 766 00 · www.mediroyal.se

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© Copyright MediRoyal Nordic AB, Sweden · 2021 · 25868DR · www.vajer.se