<span>GB</span> English – Instructions
EpiArc

**General information** · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the white textile label.

**Indications** · Lateral (tennis elbow) or medial (golfers elbow) epicondylitis.

**Pre Application** · To select the right size, take circumference measurement 5 cm below the elbow joint. Use the measurement chart to determine the right size.

**Adjustment** · The widest part of the brace should be placed over the highest position of the muscle. If you have lateral epicondylitis the pad should be placed on the upper part of the arm. If you have medial epicondylitis, place the pad on the lower side of the arm.

**Application** · Open the Velcro strap. Position the wider part of the brace with the silicone inside over the highest part of the muscle. Close the strap but do not over tighten. When you make a fist with your hand the pressure from the pad should be felt over the muscle. The brace is only effective during loading of the hand/arm. It shouldn't be used when you rest or sleep. Epicondylitis are usually caused by over extension or wrong position of the wrist. Therefore it might be a good idea to combine the brace with a wrist brace as well. This unloads the tendons and decreases the pain. For the complete assortment of wrist braces, consult www.mediroyal.se

**Removal** · Open the Velcro strap and remove the brace. Always then check for pressure marks.

**Caring instructions and precautions** · We recommend that the brace should be washed once a week. Follow the washing instructions on the label and always use the laundry bag. Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on the label.

More information can be found on www.mediroyal.se

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<span>GB</span> English – Instructions
EpiArc
<b>General information</b> · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the white textile label.
<b>Indications</b> · Lateral (tennis elbow) or medial (golfers elbow) epicondylitis.
<b>Pre Application</b> · To select the right size, take circumference measurement 5 cm below the elbow joint. Use the measurement chart to determine the right size.
<b>Adjustment</b> · The widest part of the brace should be placed over the highest position of the muscle. If you have lateral epicondylitis the pad should be placed on the upper part of the arm. If you have medial epicondylitis, place the pad on the lower side of the arm.
<b>Application</b> · Open the Velcro strap. Position the wider part of the brace with the silicone inside over the highest part of the muscle. Close the strap but do not over tighten. When you make a fist with your hand the pressure from the pad should be felt over the muscle. The brace is only effective during loading of the hand/arm. It shouldn't be used when you rest or sleep. Epicondylitis are usually caused by over extension or wrong position of the wrist. Therefore it might be a good idea to combine the brace with a wrist brace as well. This unloads the tendons and decreases the pain. For the complete assortment of wrist braces, consult www.mediroyal.se
<b>Removal</b> · Open the Velcro strap and remove the brace. Always then check for pressure marks.
<b>Caring instructions and precautions</b> · We recommend that the brace should be washed once a week. Follow the washing instructions on the label and always use the laundry bag. Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on the label.
More information can be found on www.mediroyal.se

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<span>GB</span> English – Instructions
EpiArc
<b>General information</b> · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the white textile label.
<b>Indications</b> · Lateral (tennis elbow) or medial (golfers elbow) ep

<b>LT</b>	<b>Latvijas – Instrukcija</b>
<span>EpiArc</span>	

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