#### Size chart - Available in left or right model

| Size      | S     | M     | L     | XL    |
|-----------|-------|-------|-------|-------|
| Shoe size | 35-39 | 40-43 | 41-44 | 44-48 |

© Copyright MediRoyal Nordic AB, Sweden - 2021 - 25820DR - www.vajer.se

## Mediroyal – Functional Swedish Design

Product development is a combination of ideas, material, design and know-how. At Mediroyal we have more than 20 years of experience with developing products that provides both the patient and the therapist with enhanced functionality, everyday. Swedish design is unique in many ways. Quality, performance, functionality and design are all features that make a difference. That's why we are proud to label our products with "Functional Swedish Design". Because orthoses are our passion.

# MediRoyal Malleum AFO

MediRoyal Nordic AB - Box 7052 - SE-192 07 Sollentuna - Sweden  
+46 8 506 766 00 - www.mediroyal.se

#### GB English – instructions

Mediroyal Malleum AFO

**General information** · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content.

**Indications** · The product has been designed to provide support for drop-foot syndrome.

**Contraindications** · The product should not be used for treatment of fractures in the foot or ankle.

**Pre-Application** · Check that the patient fits within the measurement chart. Note that the product is made in left or right version.

**Application** · Open all hook and loop straps. It is recommended that the patient should wear a sock under. Start with the calf section, make sure that the textile cover protects the skin. Apply the foot part and the strap over the ankle. The sole can be trimmed if necessary. The frame can be adjusted by using a heat gun. The orthosis should be worn with a shoe for the best result.

**Removal** · Open all straps and remove.

**Material content** · The orthosis is made from Polyamide. The foam lining is made from polyamide and polyester.

**Caring instructions and precautions** · The orthosis can be washed with soap and water or rinsed with alcohol. The lining can be machine washed at 40 degrees and then let air dry.

The orthosis should not be used without a shoe as it might increase the risk for slipping.

Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content before use.

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