Size chart – Knee Immo Child

Size	S	M	L
Height in cm	30	35	40
Circumf. knee cm	23–29	26–36	30–40
Circumf. calf cm	19–25	22–31	25–39

© Copyright MediRoyal Nordic AB, Sweden · 2021 · 25846DR · www.vajer.se

GB English – instructions Knee Immo Child

General information · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient suffers from allergy, check the material content on the label.

Indications · The product has been designed to support and to immobilize the knee joint.

Contraindications · Patients with instable fractures that has not been fixated should not use the product.

Pre-Application · Check that the patient fits within the measurement chart.

Application · Open all straps. In the pockets behind and on the side of the brace there are splints that should be adjusted individually to the patient. Make the adjustment and insert the splints in the pockets again.

Adjustment · The straps are adjustable and should be cut in the desired length after application.

Removal · Open all straps and remove the brace.

Material content · Check the white label on the product for material content if you are allergic.

Caring instructions and precautions · We recommend that the brace should be washed once a week during use. Follow the washing instruction on the label, use a laundry bag, close all straps and remove the splints before.

Patients with sensitive skin, diabetes or poor circulation should be observed for pressure marks. Patients with allergies should check the material content before use.

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For mer information besök www.mediroyal.se

Knee Immo Child

Mediroyal – Functional Swedish Design

Product development is a combination of ideas, material, design and know-how. At Mediroyal we have more than 20 years of experience with developing products that provides both the patient and the therapist with enhanced functionality, every day. Swedish design is unique in many ways. Quality, performance, functionality and design are all features that make a difference. That's why we are proud to label our products with "Functional Swedish Design". Because orthoses are our passion.

MediRoyal Nordic AB · Box 7052 · SE-192 07 Sollentuna · Sweden
+46 8 506 766 00 · www.mediroyal.se

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