# MediRoyal Foam Collar

MR515 Foam Collar · Gray

Height cm	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
Art.no	6 MR515-06	8 MR515-08	10 MR515-10	12 MR515-12	

MR516 Foam Collar · Black

Height cm	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
Art.no	6 MR516-06	8 MR516-08	10 MR516-10	12 MR516-12	

 [www.mediroyal.se](http://www.mediroyal.se)

## Washing instruction

 Materials foam: Polyuretan/Polyester  
 Materials cover: 100% Cotton

MediRoyal Nordic AB  
Staffans väg 6B  
SE-192 78 Sollentuna. Sweden

+46 (0)8 506 766 00  
[info@mediroyal.se](mailto:info@mediroyal.se)  
[www.mediroyal.se](http://www.mediroyal.se)

## English – instructions

**General information** - Read the instructions carefully. The product must be applied by medically trained personnel such as an occupational therapist, physiotherapist, orthopedic engineer / technician or doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the label.

**Indications** - This product is designed to support the neck after soft tissue injuries.

**Contraindications** - Unstable fractures should not be treated with this product.

**Application** - Select the right height. Open the velcro strap and apply the collar around the neck. Close the Velcro at the back.

**Remove the orthosis** - Open the Velcro at the back.

**Care and precautions** - The product can be machine washed at 40 degrees and then air dried. Patients with reduced sensitivity, sensitive skin, diabetes or poor circulation should always be monitored for pressure problems or edema. For more information visit [www.mediroyal.se](http://www.mediroyal.se)

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