English – instructions

MediRoyal Sacro-X

General information · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the backside of the white textile label on the cover for material content.

Indications · The product has been design to provide support to the lumbar and SI-joint area.

Contraindications · Patients with reduced sensibility should consult their physician before use.

Pre Application · To select the right size, take the circumferential measurement around the waist. Use the measurement chart to determine the most suitable size.

Adjustment · The brace are fitted with a sacro-pad and two plastic splints. The pad will increase SI support and the plastic splints will both provide light support and pressure displacement.

Application · Open the Velcro closure of the brace. Apply the brace over the hip and close in the front. Then grip the reinforcement strap and apply it in the front of the brace by the Velcro closure. Adjust compression to a comfortable level.

Removal · Open the Velcro straps and remove the brace.

Caring instructions and precautions · We recommend the brace should be washed once a week. Follow the washing instructions on the white label on the product, close all straps and use a laundry bag.

Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on the backside of the label on the cover.

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