MR1400

MR1406

MR1401

MR1407

MR1402

MR1408

MR1403

MR1411

MR1405

© Copyright MediRoyal Nordic AB, Sweden · 2021 · 25863DR · www.vajer.se

English – Instructions

Impact Insoles

General information · Read the instruction carefully. The product should be used to temporary provide shock absorption and support. If your condition does not improve you should consult a doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use.

Indications · The insoles have been designed to provide shock absorbing and support. Each design has a different function:

MR1400 · Heel wedges for Plantar Fasciitis
MR1401 · Heel wedges for shock absorption
MR1402 · Heel wedges with lower profile for shock absorption
MR1403 · Arch supports
MR1405/MR1406 · Fore foot wedges for pronation/supination
MR1407/MR1408 · Heel posting wedges for pronation/supination
MR1411 · Shock absorbing insole

Pre-application · Select the right size with using the measurement chart.

Application and adjustment · The product should be used in a closed shoe with laces and not in sandals. Apply the product into the shoe. If the space gets too tight you might remove the insole in the shoe to gain some more space.

Use · The insole can be used during all active time to provide shock absorption and support. For the first couple of days, it might be a good idea to gradually get used to it by wearing it for a few hours only. If your condition gets worse, stop using the product and consult a doctor.

Material content · The product is made from Polyurethane Gel Polymer.

Caring instructions and precautions · We recommend that the insoles should be washed once a week. Use mild soap and water. Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content before use. For more information visit www.mediroyal.se

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MR1411 · pörutusi leevedav sisestald

Mediroyal – Functional Swedish Design

Product development is a combination of ideas, material, design and know-how. At Mediroyal we have more than 20 years of experience with developing products that provides both the patient and the therapist with enhanced functionality, every day. Swedish design is unique in many ways. Quality, performance, functionality and design are all features that make a difference. That's why we are proud to label our products with "Functional Swedish Design". Because orthoses are our passion.

MediRoyal Nordic AB
Box 7052 · SE-192 07 Sollentuna · Sweden
+46 8 506 766 00 · www.mediroyal.se

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