
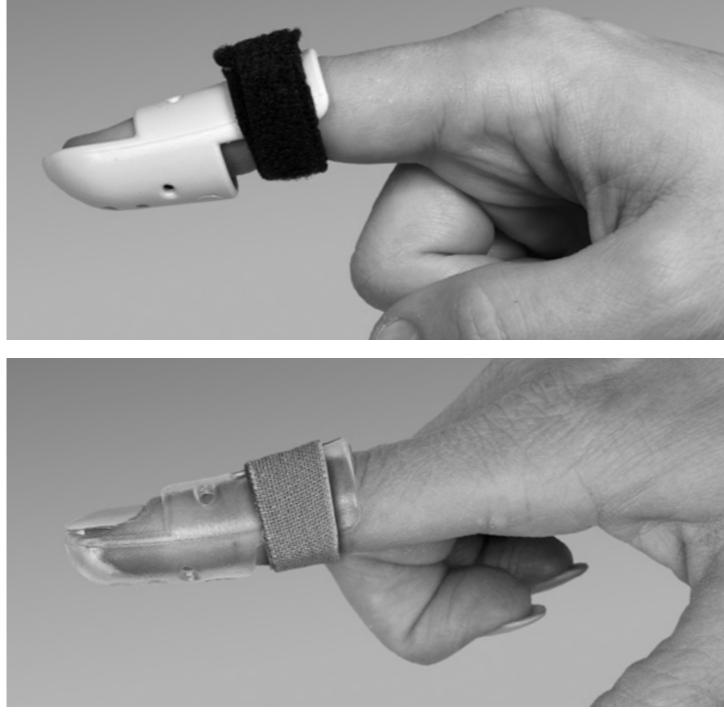
Mallet Splint & Mallet Splint Clear

MediRoyal Nordic AB
 Box 7052 · SE-192 07 Sollentuna · Sweden
 Tel +46 8 506 766 00 · www.mediroyal.se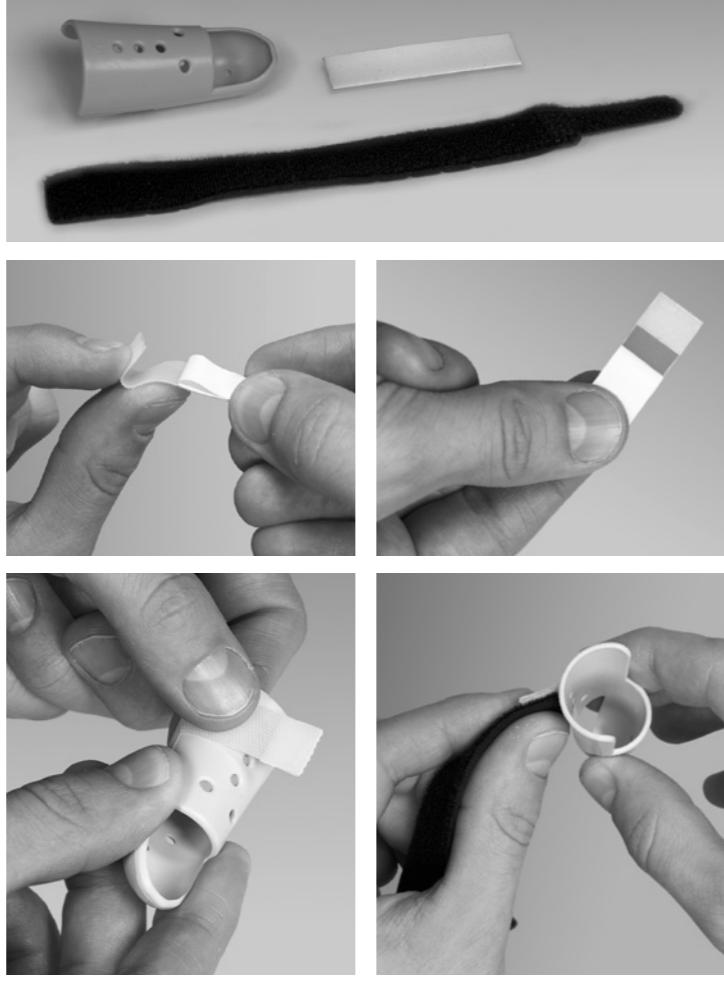

Size chart

Size	1	2	3	4	5	5,5	6	7
Circumference DIP joint mm	40-45	45-50	50-55	55-60	60-65	65-70	70-80	80-90
Length of splint mm	50	53	57	60	58	61	61	63

© Copyright MediRoyal Nordic AB, Sweden · 2023

26134DR · www.vajer.se

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GB English – instructions

MediRoyal Mallet Splint

General information · Read the instruction carefully. If you got an injury, you should consult with a doctor or therapist for an individual rehabilitation plan. If you suffer from poor circulation, reduced sensitivity, or diabetes, be observant on pressure marks or local swelling when you use the product. If your condition gets worse when using the product, discontinue the use and contact a doctor.

If the product has been fitted by a medical trained professional like occupational therapist, physical therapist, orthopedic technician or a medical doctor, please refer to them for advice and further instructions. Follow the washing instruction below. If you have material allergies you can find the material composition in the instruction. For more information, visit www.mediroyal.se

Indications · The product has been designed to provide support to the DIP joint after a mallet injury.

Contraindications · Patients with edema and circulatory problems could be a contraindication. This should be determined by the therapist or doctor.

Pre application · Use the measurement chart to select the right size. Note that the measurement chart is only a guidance to the right size and the splint should be fitted individually.

Adjustment · The splint can be adjusted if needed. Use a heat-gun and spot heat the area that needs to be adjusted. When the area is soft, shape it against an object and avoid touching it as you might burn yourself. The edges can also be trimmed with a pair of scissors.

Application · Apply the splint with the hand resting flat on a table. The splint can be secured with surgical tape or for easier removal and application using the NRX® Strap PLUS.

When using the NRX® Strap PLUS the strap will be secured with using the self-adhesive NRX® Hook. Cut a strip and fold 10 mm on the end. This will be used to attach the strap. Apply the strip on the dorsal part of the splint. Attach the NRX® Strap PLUS material and run it around the finger. The NRX1008-06 NRX® Strap Kit for twin bandages works excellent for this.

Pay attention to pressure marks or circulatory problems after a few hours of use. The splint should be used consecutively 24 hours a day for at least 6 weeks. Consult your therapists or doctor if you experience any problems during the treatment.

Removal · The splint should be removed once a day for cleaning and to check the skin condition. Rest the hand on a table, remove the splint and support the finger on the table to prevent flexion during the cleaning.

Material content · 2003/2004: Polycarbonate, 2005/2006: ABS

Caring instructions and precautions · We recommend that the splint and skin should be washed once a day. Wash with soap and water or disinfection alcohol with minimum 70%. Patients with sensitive skin, diabetes or poor circulation should check frequently for pressure marks. Patients with allergies should check the material content before use. For more information consult www.mediroyal.se

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