Size chart

Size cm	S 12,5–16	M 16–19	L 19–23

© Copyright MediRoyal Nordic AB, Sweden - 2022 - 26017DR - www.vajer.se

GB English – instructions Handy Wrist with Boa® Fit System

General information · Read the instruction carefully. If you got an injury, you should consult with a doctor or therapist for an individual rehabilitation plan. If you suffer from poor circulation, reduced sensitivity, or diabetes, be observant on pressure marks or local swelling when you use the product. If your condition gets worse when using the product, discontinue the use and contact a doctor. If the product has been fitted by a medical trained professional like occupational therapist, physical therapist, orthopedic technician or a medical doctor, please refer to them for advice and further instructions. Follow the washing instruction on the product label. If you have material allergies you can find the material composition on the label. For more information, visit www.mediroyal.se

Indications · The product has been designed for wrist instabilities caused by sprains, over-loading, carpal tunnel problems, inflammation, or post-fractures.

Contraindications · Patients with poor circulation, reduced sensitivity or diabetes should always consult their physician before use. The Boa® system provides a high degree of compression so make sure not to over-tighten.

Pre-Application · To select the right size, take the circumferential measurement around the wrist. Use the measurement chart to determine the correct size. Note that the product is made in left or right versions. The brace is fitted with an aluminum splint that can be adjusted. Remove the splint from the pocket. Adjust it by hand and then insert again.

Application · Open the Boa® system by pulling the knob outwards to release the system. Pull out the wire to make the circumference bigger. Slide the hand into the brace and the thumb out of the thumb hole. Activate the mechanism by pressing the knob and then turning it to tighten the circumference. The pressure should feel comfortable and should not be painful. Do not over-tighten.

Removal · To remove the brace, lift the knob and pull the wire, then slide out the hand.

Material composition · The material composition of the brace is printed on the care label of the product.

Caring instructions and precautions · Remove the splint before washing. The brace is recommended to hand wash with a mild detergent and let air dry. Patients with sensitive skin, diabetes or poor circulation should consult a doctor before use. For more information visit www.mediroyal.se

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Mediroyal – Functional Swedish Design

Product development is a combination of ideas, material, design and know-how. At Mediroyal we have more than 20 years of experience with developing products that provides both the patient and the therapist with enhanced functionality, everyday. Swedish design is unique in many ways. Quality, performance, functionality and design are all features that make a difference. That's why we are proud to label our products with "Functional Swedish Design". Because orthoses are our passion.

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IS Íslenska – Leiðbeiningar Handy Wrist with Boa® Fit System

IS Íslenska – Leiðbeiningar Handy Wrist with Boa® Fit System

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