## Nederlands

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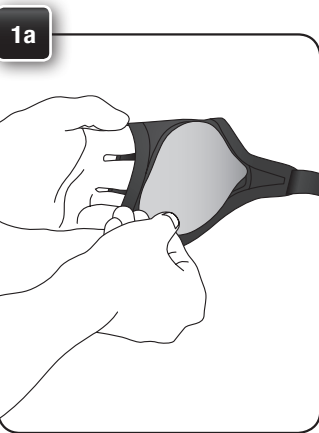
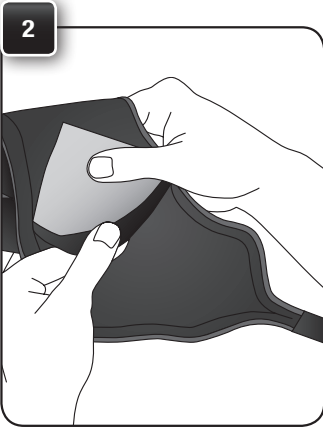
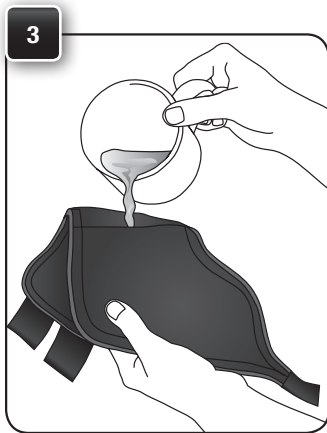
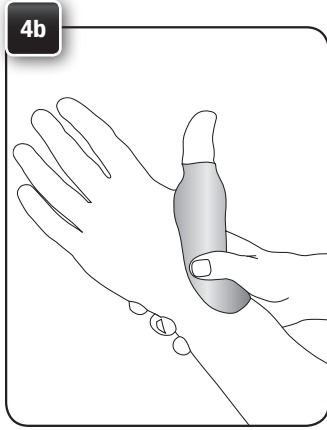
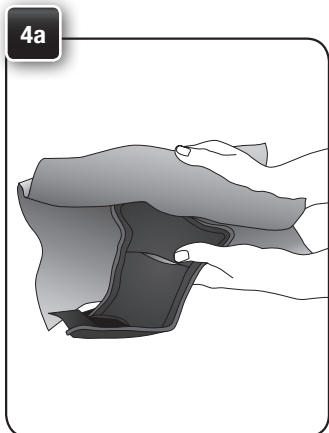
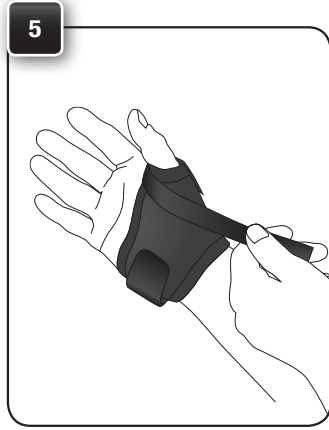
### English

Thenax Format thumb braces are designed to support and stabilize the CMC and MCP joints. The braces are made of NeoTex material with a pocket for thermoplastics. The plastic is adjusted individually by using hot water, 75-80 degrees C.

#### Instructions:

- Try the brace on without the thermoplastic piece to get the right size.
- Place the thermoplastic piece inside the pocket on the inside of the brace. Make sure that the shape fits correctly in the pocket.
- Fold the pocket slightly to open it. Pour hot water 75-80 degrees C , into the pocket. It takes usually under 1 minute for the plastic to turn soft. In needed refill the pocket with more water.
- When the plastic is soft, pour out the water and place the brace flat on a towel to absorb the moisture. Then apply it on the patient, hold it in position for 2-3 minutes until the plastic has set.
- (MR2181 only) The additional hook strap from the thumb grip can be used to stabilize and support the thumb further. Try different applications to find the most suitable support.
- If needed the plastic can be remolded by using hot water again. Simply open the pocket and start from point 3 again. The plastic can also be taken out and molded separately.

Made of NeoTex material. For more details on material content, check the backside of the label on the product.

Washing instructions: 40 degrees machine wash. See further caring instructions on the label.

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