NL

Manex Radius · Short/Long/Ease · Nederlands

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GB

Manex Radius · Short/Long/Ease · English

This brace has been designed to provide support and stability to the wrist. It can be used for post-fracture treatment, to increase wrist stability and as a night splint for carpal tunnel syndrome.

- The splint on the volar side is made of aluminum that can be bent individually. The dorsal splints are made from flexible plastic and don't need to be adjusted.
- Open the hook and loop straps around the wrist.
- Slide the brace on the wrist and position the thumb in the opening.
- Close the hook and loop straps around the wrist. Adjust the thumb strap if necessary.

If you have any allergic problems, check the material content on the backside of the label. Do not over- tighten the straps if you suffer from circulatory problems. If you are diabetic or suffer from skin problems, consult with your physician before using the brace. If your pain or problem gets worse, always consult with your therapist, physician or orthotist.

Material content: Polyester, Polyamide, Nylon, Spandex. This product is latex free.

Washing instruction: 40 degrees machine wash. See further instructions on product label.

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