General information · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content under point 8.

Indications · The product has been designed for lateral or medial ankle sprains. It can be used for chronic, temporary or prophylactic support of the ankle. The brace works best together with a stable shoe.

Contraindications · Patients with poor circulation, reduced sensibility or diabetes should always consult their physician before use. The brace should not be used for treatment of instable ankle fractures.

Pre Application · Use a sock before application of the brace. Open the two side straps on the brace. Open the Boa® system by pulling out the dial to release the lacing. The brace is fitted with plastic splints on the side. You might choose to use both or only one on the injured side. Try both to determine the best solution for you.

Application · Always apply the brace in a 90 degree position with the foot in contact with the floor. Position the tongue in the center of the brace and make sure that the upper part is aligned with the rest of the brace. Engage the mechanism by pushing the dial. Turn the dial clockwise to close the brace until you have reached a comfortable level. Depending on which side your injury is, start with the strap attached on the opposite side. Apply it over the top of the foot, cross it over the foot and attach on the loop side of the brace. Then apply the other side. When applying, try to lift the foot slightly to shorten the strap on the injured side. This will provide more stability when the strap is loaded during activity.

Adjustment · Load the brace for a few minutes. To increase stability and compression adjust the dial to tighten the brace more if needed. If you feel the need for more side stability, adjust the side straps.

Removal · To remove the brace, open the side straps. Pull out the dial to release the lacing. Slide out the foot. Assemble all the straps and close the dial system and retract the lace when the brace is not in use.

Material composition · The brace is made from the following material composition: Nylon 30%, Polyamide 65%, Polyurethane 5%.

Caring instructions and precautions · Follow the washing instruction on the label of the product. Patients with sensitive skin, diabetes or poor circulation should be observed.

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© Copyright MediRoyal Nordic AB, Sweden - 2016 - 25096DR - www.vajer.se
Boa® is a registered trademark of Boa Technology Inc.