## English – instructions

Armis Light Ankle Brace

**General information** · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content under point 8.

**Indications** · The product has been designed for lateral or medial ankle sprains. It can be used for chronic, temporary or prophylactic support of the ankle. The brace works best together with a stable shoe.

**Contraindications** · Patients with poor circulation, reduced sensitivity or diabetes should always consult their physician before use. The brace should not be used for treatment of instable ankle fractures.

**Pre Application** · Use a sock before application of the brace. Open the upper hook strap first then open the Boa® system by pulling out the wire to release the lacing. Pull out the wire until it stops. The brace is fitted with fixated plastic splints on the side. Under the foot is a friction material to provide a good grip.

**Application** · Always apply the brace in a 90 degree position with the foot in contact with the floor. Position the tongue in the center of the brace and make sure that the upper part is aligned with the rest of the brace. Engage the mechanism by pushing the dial. Turn the dial clockwise to close the brace until you have reached a comfortable level. There is a small plastic feeder that can be pulled to provide an even pressure between the lower and upper part of the brace. Then close the upper hook strap. The cross strap can be used to further enhance the stability. Apply the center of the strap to the loop surface under the foot. Criss-cross the strap around the ankle, pull it tighter on the injured side and close the strap in the back over the Boa®-system.

**Adjustment** · Load the brace for a few minutes. To adjust stability and compression tighten dial on the brace more if needed. If you feel the need for more side stability, adjust the side straps.

**Removal** · To remove the brace, open the strap and the upper hook strap. Pull out the dial to release the lacing. Slide out the foot. Assemble all the straps and close the dial system and retract the lace when the brace is not in use.

**Material composition** · The brace is made from the following material composition: Nylon 30%, Polyamide 65%, Polyurethane 5%.

**Caring instructions and precautions** · Follow the washing instruction on the label of the product. Patients with sensitive skin, diabetes or poor circulation should be observed.

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© Copyright MediRoyal Nordic AB, Sweden - 2016 - 25095DR - [www.vajer.se](http://www.vajer.se)