English – instructions

MediRoyal Manex Radial

- Take the measurement according to the measurement chart. Try the orthosis, the finger and the thumb parts individually to determine the right size. The width of the orthosis should match the width of the dorsal part of the hand.
- The thermoplastic piece can be molded either directly on the skin or inside the pocket of the orthosis:
- 2a. Skin application: If fitted on the skin directly, heat the material in hot water 75-80 degrees C. It takes usually 1-2 min before its soft. Dry the surface on a towel and then apply onto the patient. Try to keep the wrist angle as close to 0-10 degrees in extension. Pay attention to the ulna head and make sure that there is enough space to avoid pressure problems. When the material is cool, insert it into the orthosis.
- 2b. Heating inside the pocket: Place the thermoplastic inside the pocket. Fold the material slightly to open up the pocket. Use hot water 75-80 degrees C and pour it down into the pocket. Be careful not to burn your fingertips. Once the pocket is full let the orthosis stay for about 1 minute. Check that all parts of the thermoplastic are soft, if not pour more water into the pocket. When soft, pour out the water and place the orthosis on a dry towel. Fold and press to absorb moisture. Always check the temperature of the orthosis before skin application. Patients with sensitive skin should wear a protective sleeve. Try to keep the wrist angle as close to 0-10 degrees in extension. Pay attention to the ulna head and make sure that there is enough space to avoid pressure problems. The cooling takes about 3-4 minutes.
- Apply the molded orthosis on the arm. Apply the finger part and attach it onto the surface of the orthosis. Start with a low tension and adjust if necessary. Close the thumb strap on the thumb part, then slide it on to the thumb and attach onto the orthosis.
- The blue foam strap can be used to support the MCP if there is a heavy hand. It can also be used during night-time as an alternative to the finger part. Attach the double hook onto the dorsal surface. Attach one side of the foam strap onto the hook and run it under the MCP-joints and then attach onto the hook again. The foam is not elastic and will provide good support.

Replacement sets for finger and thumb parts are available:

MR2283 Finger part S-M-L, Left & Right 3-pack
MR2284 Thumb part S-M-L, Left & Right 3-pack

Washing instructions · Hand wash is recommended. Close all hook and loop straps before washing.

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© Copyright MediRoyal Nordic AB, Sweden - 2016 - 25039DR - www.vajer.se
MediRoyal Nordic AB · Box 7052 · SE-192 07 Sollentuna · Sweden