General information - Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the white textile label.

Indications - The product has been designed to provide extension assistance and stability for the upper spine for patients that have suffered vertebral compression fractures due to osteoporosis. It can also be used to decrease kyphosis and to stabilize the lumbar spine to enhance postural stability. Physical activity is an important factor for treatment of osteoporosis. EXTO™ Stable can help you to perform the activities that you want.

Contraindications - Patients with recent fractures should have them checked with their doctor before using the brace.

Pre Application - To select the right size, take the circumferential measurement around the waist. Use the measurement chart to determine the most suitable size. The brace is adjustable and the size depends sometimes on the tolerance for pressure by the individual patient.

Adjustment - The brace is made of a breathable and comfortable foam material with a hook receivable surface. It consists of the lumbar back brace part, the spinal part and the shoulder straps. Each part should be individually adjusted.

- Start with the spinal part. Open the pocket in the lower part of the brace and remove the aluminum core. This should be adjusted individually to the patient.
- Start with a modest adjustment and adjust step by step on the patient. The brace must be securely anchored over the lumbar spine to provide stability to the thoracic part. Make sure to adjust the lumbar part of the aluminum core before moving up.
- Adjust the shoulder straps and the padding. It's connected to the pull strap that is attached onto the front part of the back brace. This pull strap is also connected to the spinal part. Adjust the position of the spinal insertion pad to get a good extension as well as a good comfort. Move the pad up or down depending on how much extension you want.
- The lumbar brace should be applied low over the waist/hip to provide a good anchor. It's made from elastic, semi-elastic and non elastic materials.

Application - Once all parts have been adjusted, instruct the patient to apply the brace like a back pack. Slide one arm in, then the other one. Check that the pads are protecting the shoulder area. Secure the back brace low over the waist/hip. It must be tight in the first fitting to provide a good anchor. Pull the first two straps and attach them onto the front. The pull the two shoulder straps out from the body and slightly backwards to get an extension effect. Secure then in the front. The rear attachment pad for the shoulder straps is adjustable and can be moved down for more extension effect. Let the patient wear it for 5 minutes with both standing and sitting. Make necessary adjustments to make sure that do pressure areas will develop.

Removal - Open the two side straps and attach them further to the back to slack the compression of the shoulder straps. Then open the lumbar brace. Remove the brace like a back pack. Re-attach the hook parts when not in use, that will reduce wear and tear of the brace.

Caring instructions and precautions - We recommend that the brace should be washed once a week. Follow the washing instructions on the white label, use liquid detergent, remove the aluminum core from the back panel, close the straps and always use a laundry bag if possible.

Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on label on the product.

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