Size chart

MR1700 · LOW – Lateral Orthotic Wedge Insoles

Size	34/35	36/37	38/39	40/41	42/43	44/45	46/47

© Copyright MediRoyal Nordic AB, Sweden · 2021 · 25862DR · www.vajer.se

Mediroyal – Functional Swedish Design

Product development is a combination of ideas, material, design and know-how. At Mediroyal we have more than 20 years of experience with developing products that provides both the patient and the therapist with enhanced functionality, every day. Swedish design is unique in many ways. Quality, performance, functionality and design are all features that make a difference. That's why we are proud to label our products with "Functional Swedish Design". Because orthoses are our passion.

low

Lateral Orthotic Wedge
INSOLES

English GB
Svenska SE
Suomi FI
Íslenska IS
Eesti EE
Deutsch DE
Français FR
Italiano IT
Español ES
Português PT
Nederlands NL
Lietuvos LT
Latvijas LV
Polski PL
Česky CZ
Русский RU
Slovenski SI
Română RO
Ελληνικά EL
Malay MY

MediRoyal Nordic AB
Box 7052 · SE-192 07 Sollentuna · Sweden
+46 8 506 766 00 · www.mediroyal.se

GB English – Instructions

LOW · Lateral Orthotic Wedge

General information · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician, or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use.

Indications · The insole has been designed to correct and compensate supination of the foot. It can be used for a wide range of diagnoses that are related to supination decided by your medical professional.

Contraindications · Patients with pronation of the foot should not use this insole.

Pre-application · Select the right size with using the measurement chart.

Application and adjustment · Replace the existing insole of the shoe with the LOW insole. If the insole is too big, trim the shape by using a pair of scissors.

Use · The insole can be used during all active time. For the first couple of days, it might be a good idea to gradually get used to it by wearing it for a few hours only.

Material content · The insole is made from EVA Ethylene Vinyl Acetate, CAS Nb 24937-78-8

Caring instructions and precautions · We recommend that the insoles should be washed once a week. Use mild soap and water.

Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content before use.

For more information visit www.mediroyal.se

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