Size chart

MR1500 · MOW – Medial Orthotic Wedge Insoles

Size	34/35	36/37	38/39	40/41	42/43	44/45	46/47
Shoe length	250	255	260	265	270	275	280

© Copyright MediRoyal Nordic AB, Sweden · 2021 · 25861DR · www.vajer.se

GB English – Instructions MOW · Medial Orthotic Wedge

General information · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician, or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use.

Indications · The insole has been designed to correct and compensate pronation of the foot. It can be used for a wide range of diagnoses that are related to pronation decided by your medical professional.

Contraindications · Patients with supination of the foot should not use this insole.

Pre-application · Select the right size with using the measurement chart.

Application and adjustment · Replace the existing insole of the shoe with the MOW insole. If the insole is too big, trim the shape by using a pair of scissors.

Use · The insole can be used during all active time. For the first couple of days, it might be a good idea to gradually get used to it by wearing it for a few hours only.

Material content · The insole is made from EVA Ethylene Vinyl Acetate, CAS Nb 24937-78-8

Caring instructions and precautions · We recommend that the insoles should be washed once a week. Use mild soap and water.

Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content before use.

For more information visit www.mediroyal.se

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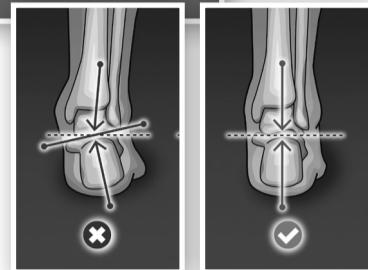
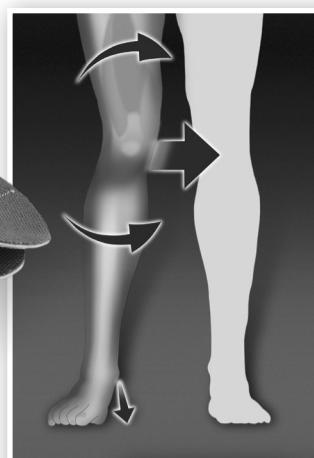
mediroyal

mow
Medial Orthotic Wedge
INSOLES

Mediroyal – Functional Swedish Design

Product development is a combination of ideas, material, design and know-how. At Mediroyal we have more than 20 years of experience with developing products that provides both the patient and the therapist with enhanced functionality, every day. Swedish design is unique in many ways. Quality, performance, functionality and design are all features that make a difference. That's why we are proud to label our products with "Functional Swedish Design". Because orthoses are our passion.

MediRoyal Nordic AB
Box 7052 · SE-192 07 Sollentuna · Sweden
+46 8 506 766 00 · www.mediroyal.se

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