## English – instructions

Mediroyal Dorsum Flexi  
Mediroyal CoTex Dorsum Stable

**General information** · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the white label.

**Indications** · The product has been designed to provide support and stability to the lumbar area.

**Contraindications** · Patients with instable fractures should use a more stable brace.

**Pre Application** · To select the right size, take the circumferential measurement around the waist. Use the measurement chart to determine the most suitable size.

**Adjustment** · The brace is fitted with four metal splints. They are located in the back in closed laminated pockets. Before application, slightly bend the splints over a table edge to fit the curvature of the back. This will provide a more comfortable support and will prevent the risk of pressure marks over the endings of the splints. Mediroyal Dorsum Flexi is also fitted with a lumbar pad for additional comfort.

**Application** · The Dorsum Flexi has a two-piece front closure. Open the Velcro straps. Always start with applying the lower one first. Then attach the two upper straps onto the front of the brace.

The CoTex Dorsum Stable has a one-piece closure. Open the Velcro straps. Apply the front by attaching the Velcro. The side straps provide individual lumbar stability.

**Removal** · Open the Velcro straps and remove the brace. Always then check for pressure marks after the removal.

**Caring instructions and precautions** · We recommend that the brace should be washed once a week. Follow the washing instructions on the white label and always use the laundry bag. Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on the white label.

More information can be found on [www.mediroyal.se](http://www.mediroyal.se)

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