General information · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor on the textile label. If the patient is allergic, check the material content on the label.

Indications · The product has been designed to provide support and immobilization of the shoulder after surgery or trauma.

Contraindications · Patients with unstable fractures should not use this product unless the fracture has been stabilized.

Pre Application · Select the right size using the sizing reference on the measurement chart.

Adjustment · The length of the pocket is important for the support of the sling. Make sure to select the right size, rather a little too long than too short. Make sure that the zipper is closed upon fitting. The strap around neck is adjustable by the hook strap. The pad around the waist is adjustable with the buckle. On top of the pad is an angled wedge. This one can be used to increase the external rotation if needed. Always remove it on the first application and install if needed.

Application · Apply the waist pad first. Close the buckle and adjust the length of the strap. Remove the wedge but keep the double hook part on the pad. The plastic buckle of the shoulder sling should be worn on the outside. If needed just swap places by removing the Velcro piece. Make sure to run the strap through the D-ring, then into the buckle to secure. The loose end can be fixed by the removable hook piece. Open the hook straps on the sling and slide the arm into the pocket. Position the arm so it fits into the pocket, the cushions should provide comfort to the side of the elbow joint. Adjust and secure the hook strap around the neck. Position the arm sling pocket onto the waist pad with the double hook.

The zipper can be used to extend the arm during long term immobilization. Make sure to open completely controlled and only under medical supervision from a therapist or doctor.

Removal · Open the straps and buckles and remove the pad and the sling.

Caring instructions and precautions · We recommend that the arm sling should be washed once a week during long term immobilization. The pad can be washed as well but remove the foam insert first. Follow the washing instructions on the white label on the product, close all straps and use a laundry bag if possible.

Patients with sensitive skin, diabetes or poor circulation should be observed.

Patients with allergies should check the material content on label.

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© Copyright MediRoyal Nordic AB, Sweden - 2016 - 25077DR - www.vajer.se