English – instructions

Wrist Supports

General information · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the backside of the white textile label.

Indications · The product has been designed to provide support to the wrist.

Contraindications · Patients with instable fractures should not use this brace.

Pre Application · To select the right size, take the circumferential measurement around the wrist. Use the measurement chart to determine the most suitable size. Note that the product made in left or right versions.

Adjustment · The brace is fitted with a splint which can be removed through the opening in the end of the pocket. The splint is made of a thermo-mouldable plastic that can be adjusted to the patient by using hot water. Try the splint on the patient. Determine where it needs to be adjusted. Immerse the splint into hot water, minimum 85-90 degrees Celsius. Let it stay immersed for at least 1 minute. Remove it and change the splint into desired position and immediately cool in under running cold tap water. The splint fixates then in the new desired position. If you need to adjust more, simply repeat the process.

Application · Open the Velcro straps of the brace. Position the wrist so the splint is positioned inside the palm. Close the straps and readjust the compression.

Removal · Open the Velcro straps and remove the brace. Close all straps when the brace is not in use.

Material content · Check the backside of the white label for material content.

Caring instructions and precautions · We recommend that the brace should be washed once a week. Follow the washing instructions on the white label on the product, close all straps and use a laundry bag.

Patients with sensitive skin, diabetes or poor circulation · Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on the backside of the label.

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© Copyright MediRoyal Nordic AB, Sweden - 2016 - 24916DR - www.vajer.se