English – instructions

Dr Med Wrist Brace with Boa® system

- General information** · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content under point 7.
- Indications** · The product has been designed wrist instabilities caused by sprains, over-loading, carpal tunnel problems, inflammation or post-fractures. It should be worn during activity.
- Contraindications** · Patients with poor circulation, reduced sensibility or diabetes should always consult their physician before use. The Boa® system provides a high degree of compression. Make sure not to over-tighten.
- Pre Application** · The brace is fitted with an aluminium splint that can be adjusted. Remove the splint from the pocket. Adjust it by hand.
- Application** · Open the Boa® system by pulling the knob outwards to release the system. Pull out the wire to make the circumference bigger. Slide the hand into the brace and the thumb out of the thumb hole. Activate the mechanism by pressing the knob and then turning it to tighten the circumference. The pressure should feel comfortable and should not give any pain. Do not over-tighten.
- Removal** · To remove the brace, lift the knob and pull the wire. Open the buckle and remove the brace.
- Material composition** · The brace is made from:
50% Polyester, 30% Polyamide, 20% Nylon
- Caring instructions and precautions** · Remove the splint before washing. The brace is recommended to hand wash with a mild detergent and let air dry. Patients with sensitive skin, diabetes or poor circulation should consult a doctor before use.

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CE MediRoyal Nordic AB · Box 7052 · SE-192 07 Sollentuna · Sweden