
Mediroyal Multi Finger Splint


CE MediRoyal Nordic AB
 Box 7052 · SE-192 07 Sollentuna · Sweden
 Tel +46 8 506 766 00 · www.mediroyal.se

Extension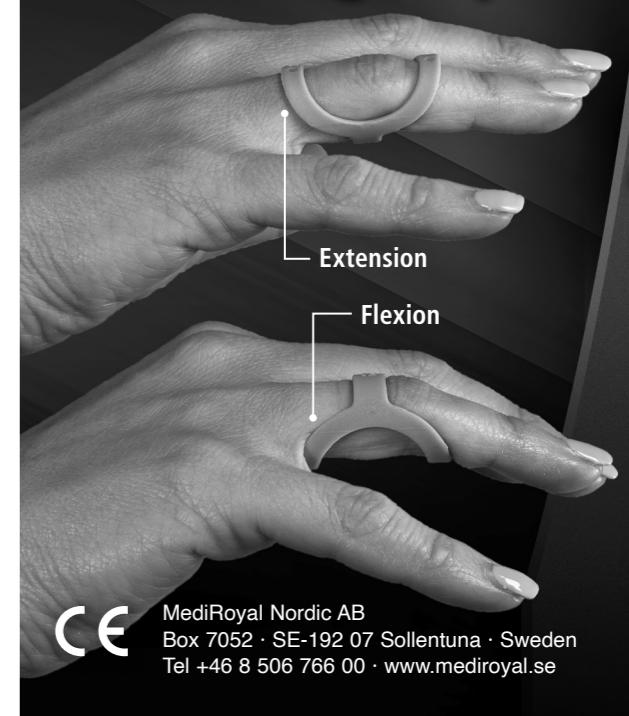


Size	Circumf. PIP joint mm	Length of splint mm
1	42–43	26
2	44–46	29
3	46–50	31
4	50–55	33
5	55–58	33
6	58–60	35
7	60–62	39
8	62–65	39
9	65–68	40
10	68–71	43
11	71–74	43
12	74–77	45
13	77–81	45
14	81–84 45	

Flexion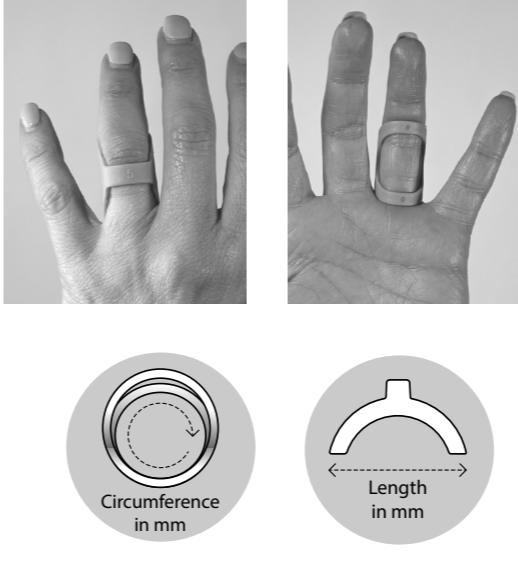

© Copyright Mediroyal Nordic AB, Sweden - 2022
 26046DR · www.vajer.se

A. Trigger Finger/Thumb

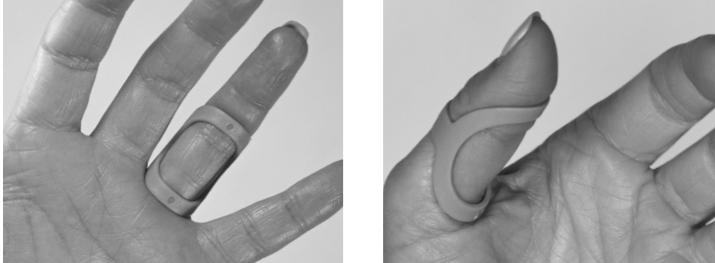B. Swan Neck Deformity



C. Boutonnière Deformity

D. Mallet Finger

E. Injuries of instabilities to the collateral ligaments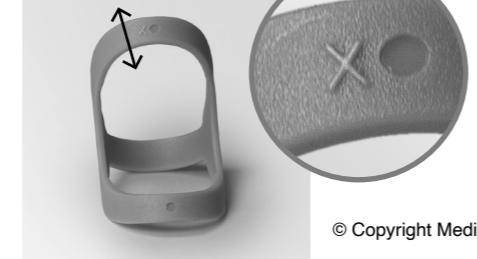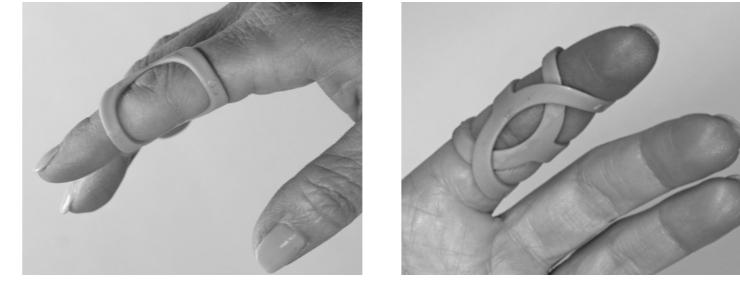

GB English – instructions Mediroyal Multi Finger Splint

General information - Read the instruction carefully. If you got an injury, you should consult with a doctor or therapist for an individual rehabilitation plan. If you suffer from poor circulation, reduced sensibility, or diabetes, be observant on pressure marks or local swelling when you use the product. If your condition gets worse when using the product, discontinue the use and contact a doctor. If the product has been fitted by a medical trained professional like occupational therapist, physical therapist, orthopedic technician, or a medical doctor, please refer to them for advice and further instructions. Follow the washing instruction on the product label. If you have material allergies, you can find the material composition on the label. For more information, visit www.mediroyal.se

Indications - The product has been designed to provide support to the DIP and PIP finger joints with problems like trigger finger, swan neck deformity, boutonnière deformity, mallet finger, instability or injuries to the collateral ligaments of the finger.

Contraindications - Patients with unstable fractures should not use this splint as a replacement for casting. Patients with edema and circulatory problems could be a contraindication. This should be determined by the therapist or doctor.

Pre application - Use the measurement chart to select the right size. Note that the measurement chart is only a guidance to the right size and the splint should be fitted individually for an optimal function. One of the sides of the splint is a little wider than the other part. This part is marked with a plus-sign on the outside.

Adjustment - The splint can be adjusted if needed. Use a heat-gun and spot heat the area that needs to be adjusted. The plastic composition needs a temperature between 140–160 degrees C to become adjustable. When the area is soft, shape it against an object and avoid touching it as you might burn yourself. The splint can then be cooled with water.

Application - The splint can be used different ways to provide support. See the pictures for the different applications.

A. Trigger finger or trigger thumb

B. Swan neck deformity

C. Boutonnière deformity

D. Mallet finger

E. Injuries or instabilities to the collateral ligaments.

Pay attention to pressure marks or circulatory problems if the splint is too tight. The splint must be adjusted or replaced with a larger size. If the application involves two splints in combination, note that the overlapping splint needs to be a larger size.

Removal - Slide the splint off the finger after use.

Material content - The splint is made from polyamide and nylon.

Caring instructions and precautions - We recommend that the brace should be washed once a week. Wash with soap and water or disinfection alcohol with minimum 70%. Patients with sensitive skin, diabetes or poor circulation should check frequently for pressure marks. Patients with allergies should check the material content before use. For more information consult www.mediroyal.se

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