**General information** · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the white textile label.

**Indications** · The product has been design to support the lumbar part of the spine after surgery, general instabilities, spinal compression fractures or during painful conditions.

**Contraindications** · Patients with instable fractures should consult a doctor before using any back support.

**Pre Application** · To select the right size, take the circumferential measurement around the waist. Use the measurement chart to determine the most suitable size.

**Adjustment** · The brace is fitted with a lumbar support panel made of plastic. This panel can be removed by opening the zipper on the lower side of the lumbar pocket. The panel can be adjusted by using a heat gun and can also be grinded. The pulling strings in the back can be adjusted in the front where the hook pieces are attached. The lumbar part can be detached from the back brace part by opening the press buttons in the back.

**Application** · SpinoQuick® has been designed to provide good lumbar support without having to use excessive power. This makes it suitable for patients with weak hands and arms. The front closure has plastic reinforcements integrated to make it more stable during the application. The handles are design to make the application easier. Apply the brace over the lumbar region and close in the front. Release the pulling straps from the side and attach them over the front. Adjust compression to a comfortable level.

**Removal** · Open the pulling straps first. Then open the main closure and remove the brace.

**Caring instructions and precautions** · We recommend that the brace should be washed once a week. Follow the washing instructions on the white label on the product, close all straps, remove the lumbar plastic insert from the pocket and use a laundry bag to protect the zipper and straps.

Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on the white label.

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