GB English – Instructions NRX® Tendens Wrist

General information · Read the instruction carefully. If you got an injury, you should consult with a doctor or therapist for an individual rehabilitation plan. If you suffer from poor circulation, reduced sensitivity, or diabetes, be observant on pressure marks or local swelling when you use the product. If your condition gets worse when using the product, discontinue the use and contact a doctor.

If the product has been fitted by a medical trained professional like occupational therapist, physical therapist, orthopedic technician or a medical doctor, please refer to them for advice and further instructions. Follow the washing instruction below. If you have material allergies you can find the material composition in the instruction. For more information, visit www.mediroyal.se

Indications · The product has been designed to provide ulnar and radial support over the wrist and to support the opening-grip of the thumb.

Contraindications · Patients with instable fractures should not use this brace as an alternative for a cast.

Pre Application · To select the right size, take the circumferential measurement around the wrist. Use the measurement chart to determine the correct size. Note that the product is made in a left or right version.

Adjustment · The brace is fitted with flexible steel splints on the sides. They should be adjusted for optimal support over the wrist. The splint can be removed from the bottom of the pocket. On the dorsal side of the brace is an NDX material with a friction surface on the inside. When extending the wrist into dorsal extension it assists in slowing down the movement and the extra strap provides stability. The lower hook strap has a fleece tongue that can be cut to expose more hook if needed.

Application · Open the hook straps but keep the thumb part closed. Apply the brace over the thumb and then apply the wide strap around the wrist by attaching the hook part onto the main body. Pay attention so the hook part is applied over base skin. Then apply the lower hook strap. If you need to expose more hook, loosen the fleece tongue and trim it to the desired length. After that, attach the wide elastic strap around the wrist. Pay attention to the stability around the wrist. For more control, adjust the splints.

Removal · Open all the hoop parts and slide off the brace. Close all straps when the brace is not in use.

Material content · Check the care label for the material composition if you have material allergies.

Caring instructions and precautions · We recommend that the brace should be washed once a week. Follow the washing instructions on the care label, close all straps and use a laundry bag. The splints should be removed. Patients with sensitive skin, diabetes or poor circulation should check frequently for pressure marks. Patients with allergies should check the material content on the care label before use. For more information consult www.mediroyal.se

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