



QuickFit™ Lacer
by CLICK

Replacement Dial Kit Size: XS/SM/MD

RP0200-130-05






For help and video instructions:
clickmedical.co/quickfit-lacer

Revised 2019

Kit Contents:

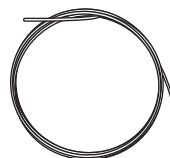

1x Replacement
Lace, 1.5m



1x Replacement
BOA® Reel

Compatible with:

QuickFit™ Lacer XS: OK0180-130-05

QuickFit™ Lacer SM: OK0200-130-05

QuickFit™ Lacer MD: OK0220-130-05

Advisory



Must be installed by a trained professional.
Lace will wear. At minimum, inspect monthly.
Overtightening could reduce circulation.
Do not use on patients with nerve issues.
Keep lace away from sharp edges.

See advisories in other languages: clickmedical.co/advisories



WARRANTY:

www.clickmedical.co/terms/#warranty

U.S. Customers:

Contact Click Medical directly


Help@ClickMedical.co

Help Line: +1 970 670 7012


International Customers:

Please contact your local distributor

- 1 Pull reel open to release tension.

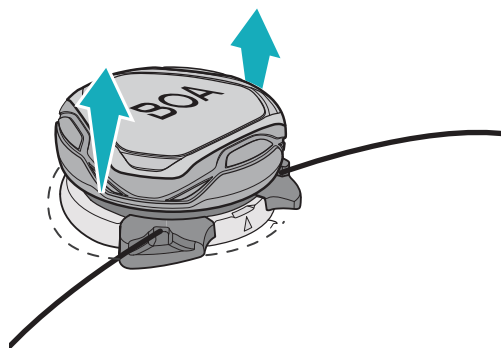


- 2 a. Insert flat-head screwdriver into the tab with an arrow indicator.
b. Use tool to pry reel upward until cartridge pops out of bayonet.

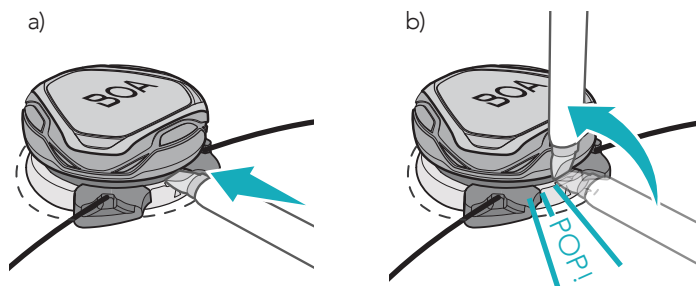


Note: Tab is located on top of mounting
plate between the lace ports.

- 3 Snip lace to remove old reel and lace from the system.

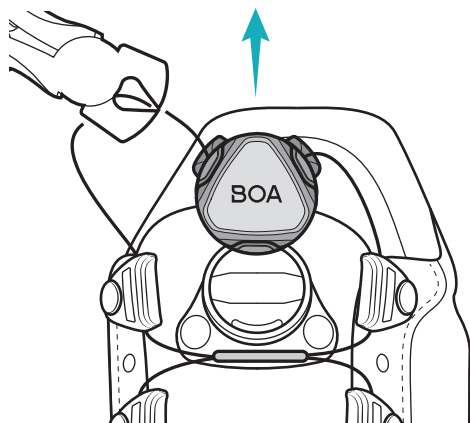


- 4 Feed new lace through all the guides.

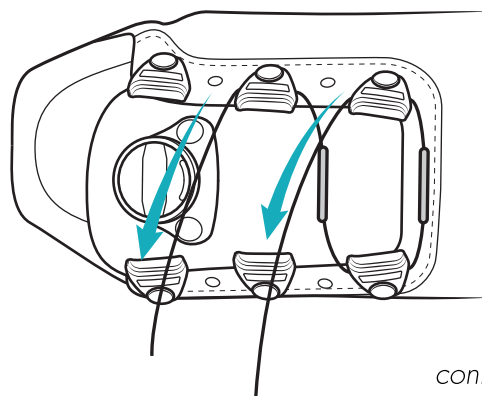


continued: ➡

→ continued:

- 5 Take new reel and press down to close it.

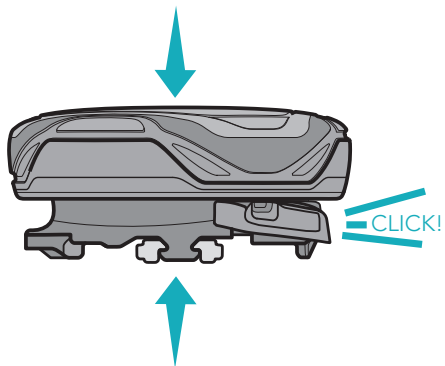


- 6 Turn until the two arrows on the back of the reel align.

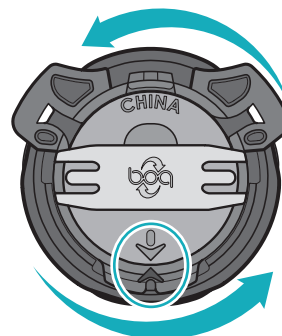


- 7 Attach lace to reel:


a) Feed right lace through through right lace port.

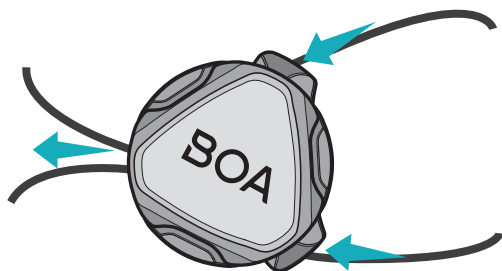


b) Determine lace length needed. Leave a maximum of 15cm of lace capacity per lace port.

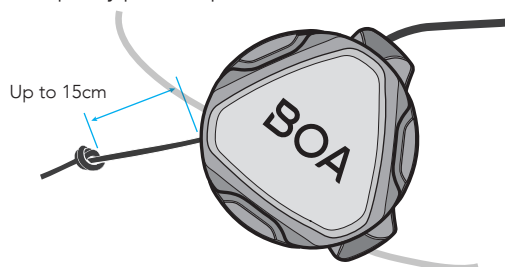

c) Tie off with a single overhand knot. Cut tail to 4-7mm past knot.

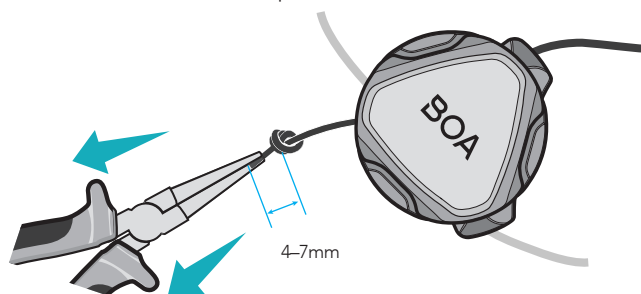


d) Pull knot into port. Repeat for opposite side of reel.

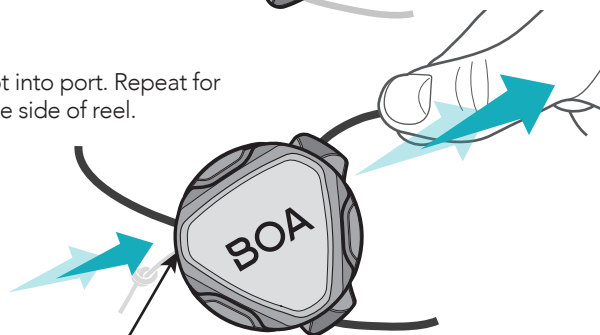


Note: Knot and tail should not be visible when the lace is seated properly.

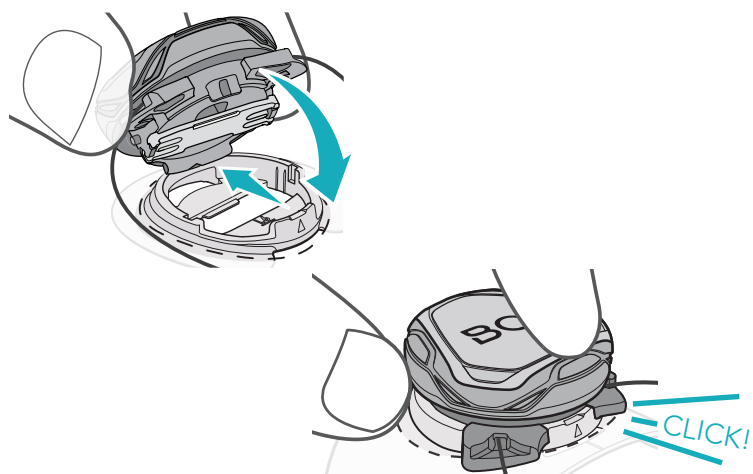

- 8 Snap reel fully into the mounting plate.

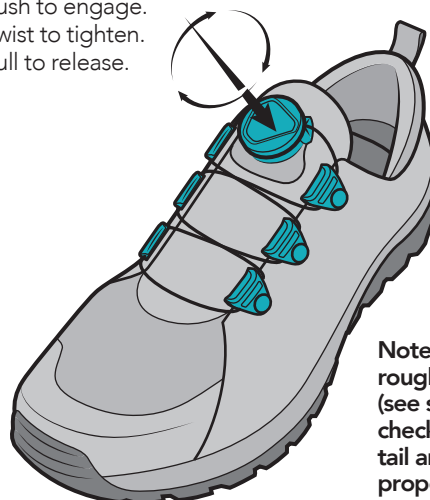

- 9 Cycle system to test for smooth function:
Push to engage.
Twist to tighten.
Pull to release.



Note: If system feels rough, remove reel (see steps 1-2) and check that knot and tail are seated properly (see step 7).

MD		MDSS GmbH. Schiffgraben 41 30175 Hannover Germany	
EC	REP		
CH	REP	MDSS CH GmbH. Laurenzenvorstadt 61 5000 Aarau Switzerland	UK REP MDSS-UK RP LIMITED 6 Wilmslow Road, Rusholme M14 5TP Manchester United Kingdom
		Click Medical, LLC 1205 Hilltop Parkway, W101 Steamboat Springs, CO, 80487, USA • 1-970-670-7012	

Revised 2022