




Clavicle Straps

Single Patient Use

Rx Only

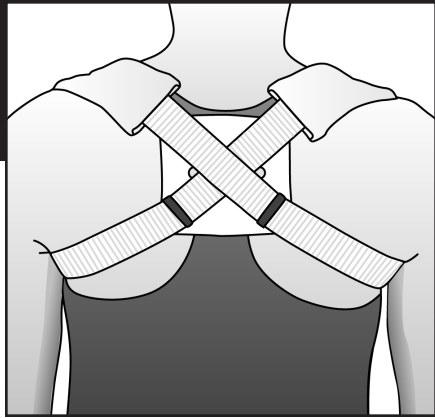

IMPORTANT INFORMATION

Please read carefully and completely before applying.

INTENDED USE

Clavicular Splints or Clavicle Straps are most often used when a patient has a broken clavicle or 'collar' bone. The product is applied to the patient so that it gives support to the broken bone and holds it in place. Clavicle Straps are commonly utilized for clavicle fracture stabilization.

INDICATIONS

Clavicle fractures; Grade II and III acromioclavicular separations; Osteolysis of the clavicle; AC degenerative joint pain; Mechanical cervical disease, postural dysfunction, Sternoclavicular dislocations.

CONTRAINDICATIONS

Patients with poor peripheral circulation, diabetes or decrease in skin sensitivity should not use this product.

PRECAUTIONS

These products are to be fitted initially by a health-care professional that is familiar with the purpose for which they are used. The healthcare professional is responsible for providing wearing instructions and precautions to other.

SIZING



Measure the distance from the center of the back, under the axilla, around the shoulder and back to the center of the back.

SIZING GUIDE

INFANT	13" x 25"
CHILD	26" x 33"
SMALL	34" x 38"
MEDIUM	39" x 42"
LARGE	43" x 47"
X-LARGE	48" x 60"

INSTRUCTIONS FOR USE

1. Center the back clavicle pad over the scapula of the unaffected side.
2. Tighten straps until snug on unaffected side.
3. Apply second strap to the affected side and pull tightly until the back pad is centered between the shoulder blades and shoulders are securely positioned back.
4. Place end of strap through buckle or attach hook and loop closure to make necessary adjustments until secure fit is achieved.

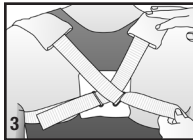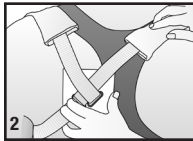

Manufactured by DeRoyal
200 DeBusk Lane
Powell, TN 37849 U.S.A.
888.938.7828
865.938.7828


Part #0-1825
©2007 DeRoyal. All Rights Reserved.
Revised 12/07



Clavicle Straps

Single Patient Use

Rx Only

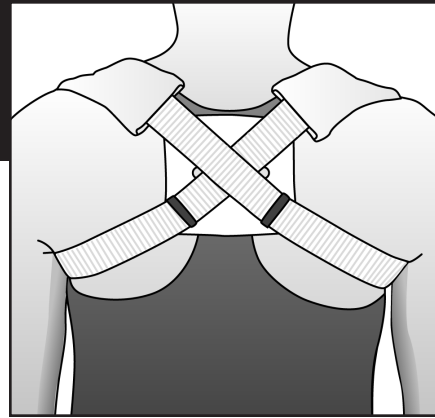

IMPORTANT INFORMATION

Please read carefully and completely before applying.

INTENDED USE

Clavicular Splints or Clavicle Straps are most often used when a patient has a broken clavicle or 'collar' bone. The product is applied to the patient so that it gives support to the broken bone and holds it in place. Clavicle Straps are commonly utilized for clavicle fracture stabilization.

INDICATIONS

Clavicle fractures; Grade II and III acromioclavicular separations; Osteolysis of the clavicle; AC degenerative joint pain; Mechanical cervical disease, postural dysfunction, Sternoclavicular dislocations.

CONTRAINDICATIONS

Patients with poor peripheral circulation, diabetes or decrease in skin sensitivity should not use this product.

PRECAUTIONS

These products are to be fitted initially by a health-care professional that is familiar with the purpose for which they are used. The healthcare professional is responsible for providing wearing instructions and precautions to other.

SIZING



Measure the distance from the center of the back, under the axilla, around the shoulder and back to the center of the back.

SIZING GUIDE

INFANT	13" x 25"
CHILD	26" x 33"
SMALL	34" x 38"
MEDIUM	39" x 42"
LARGE	43" x 47"
X-LARGE	48" x 60"

INSTRUCTIONS FOR USE

1. Center the back clavicle pad over the scapula of the unaffected side.
2. Tighten straps until snug on unaffected side.
3. Apply second strap to the affected side and pull tightly until the back pad is centered between the shoulder blades and shoulders are securely positioned back.
4. Place end of strap through buckle or attach hook and loop closure to make necessary adjustments until secure fit is achieved.

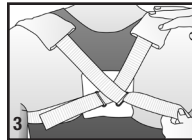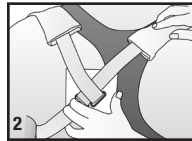

Manufactured by DeRoyal
200 DeBusk Lane
Powell, TN 37849 U.S.A.
888.938.7828
865.938.7828

Part #0-1825
©2007 DeRoyal. All Rights Reserved.
Revised 12/07