ORTOSI PUCCI® EZE PER LA MANO

IMPORTANTE

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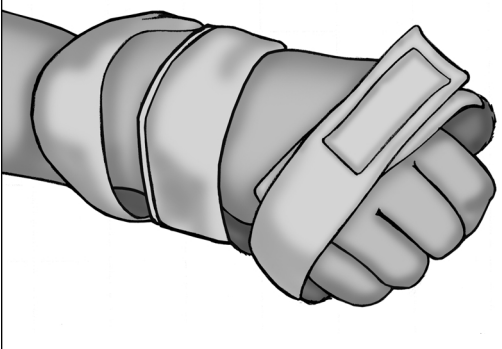
PUCCI® EZE HÅNDORTOSE

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PUCCI® EZE HANDORTOS

These products are to be fitted initially by a healthcare professional who is familiar with the purpose for which they are used. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare professionals, care providers involved in the patient's care, and the patient. Consult your medical professional immediately if you experience sensation

PRECAUTIONS

1. The Pucci EZE Hand Orthosis is supplied with a small and large hand roll. The orthosis can be used without a hand roll, or with either the small or large hand roll. **Note:** Determine the angle of the wrist position. Form the orthosis to conform to the wrist. **Note:** Without a hand roll, carefully open the fingers and position the palmar section of the orthosis under the fingers. Fingers, wrist and fingers. Fasten the hook and loop closures over the forearm, wrist and fingers. Check that they are secure and allow adequate circulation. **Note:** Check the fit of the entire orthosis, and adjust, if necessary, to ensure proper fit. The wrist portion can be bent to fit an increased range of motion position.
3. The hand rolls may be added to the Pucci EZE Orthosis for static progressive positioning, to increase range of motion. Insert the hand roll into the hand roll cover. **Note:** Insert the loop strap on the thumb side of the orthosis through the opening in the hand roll. Slide the side of the orthosis through the opening in the hand roll. Slide the hand roll onto the palmar section of the orthosis.
4. Apply the orthosis with the fingers wrapped around the hand roll.
5. Once the orthosis is fitted, check the fit at increasingly longer periods of time, starting with 15 minutes on and off, but no longer than seven hours on at any one time or follow your facilities protocol.
6. Recheck the fingers and thumb for desired position over the palmar section or hand roll. Assess fingers for adequate circulation. If circulation is impaired, remove orthosis immediately.

DIRECTIONS FOR USE

1. The Pucci EZE Hand Orthosis is supplied with a small and large hand roll. The orthosis can be used without a hand roll, or with either the small or large hand roll. **Note:** Determine the angle of the wrist position. Form the orthosis to conform to the wrist. **Note:** Without a hand roll, carefully open the fingers and position the palmar section of the orthosis under the fingers. Fingers, wrist and fingers. Fasten the hook and loop closures over the forearm, wrist and fingers. Check that they are secure and allow adequate circulation. **Note:** Check the fit of the entire orthosis, and adjust, if necessary, to ensure proper fit. The wrist portion can be bent to fit an increased range of motion position.

CONTRAINDICATIONS

Severe or fixed contractures; excessive edema or swelling; excessive pain.

INDICATIONS

Mild to moderate contractures of the wrist, hand or fingers. Contracted fingers, hand and wrist.

INTENDED USE

This product is intended to provide continuous therapy to the contracted fingers, hand and wrist.

IMPORTANT

Please read all instructions, warnings and precautions before use.

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REPLACING COVERS

Remove the cover and fasten all hook and loop closures. Place in mesh laundry bag. Machine wash on gentle cycle with mild detergent and warm water. Use no bleach or fabric softener. Rinse in cold water and air dry. Covers can be washed but not sterilized.

CARE INSTRUCTIONS

Clean the frame with a mild detergent and water. Cover washing instructions: Remove the cover and fasten all hook and loop closures. Place in mesh laundry bag. Machine wash on gentle cycle with mild detergent and warm water. Use no bleach or fabric softener. Rinse in cold water and air dry. Covers can be washed but not sterilized.

DIRECTIONS FOR USE

The Pucci EZE Hand Orthosis is supplied with a small and large hand roll. The orthosis can be used without a hand roll, or with either the small or large hand roll. **Note:** Determine the angle of the wrist position. Form the orthosis to conform to the wrist. **Note:** Without a hand roll, carefully open the fingers and position the palmar section of the orthosis under the fingers. Fingers, wrist and fingers. Fasten the hook and loop closures over the forearm, wrist and fingers. Check that they are secure and allow adequate circulation. **Note:** Check the fit of the entire orthosis, and adjust, if necessary, to ensure proper fit. The wrist portion can be bent to fit an increased range of motion position.

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