**MOLLA PER DITA**

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
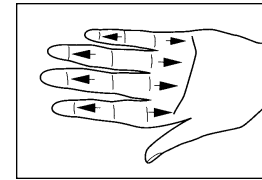
Single Patient Use

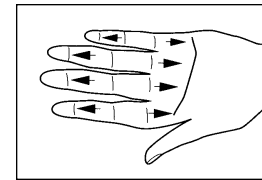**CARE INSTRUCTIONS**  
Splint can be cleaned by hand washing with a nonabrasive detergent. Air dry before wearing.

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Measuring for size: To determine splint length, measure involved finger as indicated in the diagram.

These products are to be fitted initially by a healthcare professional who is familiar with the purpose for which they are used. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare professionals, care providers involved in the patient's care, and the patient. If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.

**PRECAUTIONS**

The Finger Spring fits from the head of the proximal interphalangeal joint (PIP) to increase PIP extension while allowing active flexion.  
The following information should be read prior to use of this device.  
**IMPORTANT**  
To secure the splint, wrap the strap at the distal dorsal pad, around the volar surface of the proximal interphalangeal joint. To fit the spring, slide splint onto finger so the proximal pad rests over the volar aspect of the metacarpal head and the coils lie lateral to excess strapping

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©2003 DeRoyal. All Rights Reserved.