See Instructions for Use
Vea las instrucciones de uso
Voir les instructions d'utilisation
Gebrauchsanweisung beachten
Vedere le istruzioni per l'uso
Leia as instruções de utilização
Zie gebruiksaanwijzing
Se brugsvejledningen
Katso käyttöohjeita
Se bruksanvisning

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
Manufacturer
DeRoyal
200 DeBusk Lane
Powell, TN 37849 U.S.A.
888.938.7828

MediRoyal Nordic AB
Staffans väg 6 B, S-192 07 Sollentuna
SWEDEN +46 8 506 766 00 www.mediroyal.se

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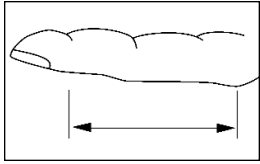
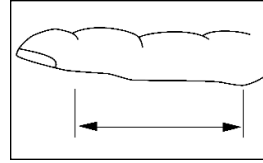
ESPAÑOL

Single Patient Use

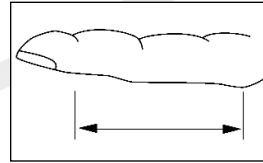CARE INSTRUCTIONS
Split can be cleaned by hand washing with nonabrasive detergent. Air dry before wearing.

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Measuring for size: To determine split length, measure involved finger use of the finger pad.

PRECAUTIONS

- 1. To fit the spring, slide split onto finger so the proximal pad rests over the distal lateral to the proximal phalanx and the coils lie lateral to the proximal interphalangeal joint.
2. The distal dorsal pad should be proximal to the proximal interphalangeal joint.
3. To secure the split, pull the strap through the D-ring to hold the split on the proximal phalanx.
4. Place the tab of hook material close to the D-ring and fold the strapping down onto it.
5. Cut excess strapping.

INTENDED USE

The Finger Spring fits from the base of the proximal phalanx to the distal interphalangeal joint. The coil springs, at the lateral aspect of the proximal interphalangeal joint (PIP), act to increase PIP flexion while allowing active extension. The distal phalanx is free to allow functional use of the finger pad.

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©2003 DeRoyal. All Rights Reserved.