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**CARE INSTRUCTIONS**

Splint can be cleaned by hand washing with a non-abrasive detergent. Air dry before wearing.

These instructions apply to product numbers: 504.

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**INDENDED USE**

This finger spring fits from the proximal phalanx to the distal interphalangeal (DIP) joint. It is used to assist the proximal interphalangeal joint (PIF) into increased extension.

To determine splint length, measure involved finger as indicated in the diagram.

**ENGLISH**

**IMPORTANT**

Read completely before applying.

**DIRECTIONS FOR USE**

**1a.** Hold the spring at the proximal and distal pads. **Note:** the proximal pad has the white plastic covered wire attached to it.

**2a.** Slide splint onto finger so the proximal pad rests over the volar surface of the proximal phalanx and the distal pad rests at the distal interphalangeal joint (DIP).

**1b.** To increase the force of extension, spread the volar pads apart before placing on the finger.

**2b.** To decrease the force of extension, pinch the volar pads together before placing on the finger.

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**Rx Only**

**Caution: Federal U.S.A. law restricts this device to sale or use by or on the order of a physician (or properly licensed practitioner).**

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©2003 DeRoyal. All Rights Reserved.