**MOLLA PER DITA**

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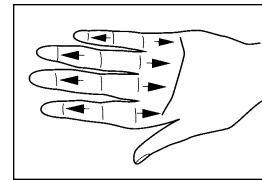
**FINGER SPRING**  
 Proximal Interphalangeal Extension Assist

**IMPORTANT**  
 The following information should be read prior to use of this device.

**INTENDED USE**  
 The Finger Spring fits from the head of the metacarpal to the distal interphalangeal joint. It is used to assist the proximal interphalangeal joint (PIP) into increased extension. It has a minimal effect on metacarpophalangeal (MP) extension. Measuring for size: To determine split length, measure involved finger as indicated in the diagram.


AA	2 1/4"	5,71 cm
A	2 5/8"	6,67 cm
B	3"	7,62 cm
C	3 3/8"	8,57 cm
D	3 3/4"	9,53 cm

**DIRECTIONS FOR USE**  
 1. Select the correct size. To fit the spring, slide split onto finger so the proximal pad rests over the volar aspect of the metacarpal head and the distal pad rests at the distal interphalangeal joint (DIP).  
 2. To adjust spring tension, there are two options:  
 a. To increase the force of extension, spread the volar pads apart before placing on the finger.  
 b. To decrease the force of extension, pinch the volar pads together before placing on the finger.  
 PRECAUTIONS  
 These products are to be fitted initially by a healthcare professional who is familiar with the purpose for which they are used. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare professionals, care providers involved in the patient's care, and the patient. If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.

Single Patient Use

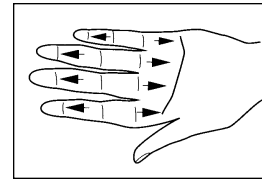**CARE INSTRUCTIONS**  
 Split can be cleaned by hand washing with a nonabrasive detergent. Air dry before wearing.

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©2003 DeRoyal. All Rights Reserved.