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Manufacturer
DeRoyal
200 DeBusk Lane
Powell, TN 37849 U.S.A.
888.938.7828

Part #1-601G
Revision C (5/03)
©2003 DeRoyal. All Rights Reserved.

FINGER SPLINT, LOW TENSION FLEXION
FÉRULA PARA DEDO PARA FLEXIÓN DE BAJA TENSIÓN
ATTELLE DE DOIGT, POUR FLEXION À FAIBLE TENSION
FINGERSCHIENE, FLEXION MIT NIEDRIGER SPANNUNG
STECCA PER DITA, FLESSIONE A BASSA TENSIONE
TALA PARA DEDO, FLEXÃO DE BAIXA TENSÃO
VINGERSPALK, FLEXIE MET LAGE SPANNING
FINGERSKINNE, LAVSPÆNDINGS FLEKSJON
SORMEN KOUKISTUSLASTA, LIEVÄ JÄNNITYS
FINGERSKENA FÖR FLEXION MED LÅG SPÄNNING

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FINGER SPLINT, LOW TENSION
IMPORTANT
Read completely before applying.
This splint fits from the head of the proximal phalanx to the distal interphalangeal (DIP) joint. This splint provides a light tension at the proximal interphalangeal (PIP) joint, to position the PIP joint in a flexed position. The distal phalanx is free to allow functional use of the finger pad.
To determine splint length, measure involved finger as indicated in the diagram.
AA 1 1/4" 3.2 cm
A 1 1/2" 3.8 cm
B 1 3/4" 4.4 cm
C 2 1/4" 5.7 cm
D 2 3/4" 7.0 cm

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Adjusting the Splint
1b. Slight adjustments in the degree of flexion can be made.
2b. Bend lateral wires, equally, as indicated in the illustration.
3b. Avoid rotating the pads of the splint.
Splint can be cleaned by hand washing with a non-abrasive detergent. Air dry before wearing.
These instructions apply to product numbers: 601.
Single Patient Use

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CARE INSTRUCTIONS

Splint can be cleaned by hand washing with a non-abrasive detergent. Air dry before wearing.
These instructions apply to product numbers: 601.
Single Patient Use

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