**STECCA PER DITA****IMPORTANTE**

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Rx Only

Caution: Federal U.S.A. law restricts this device to sale or use by or on the order of a physician (or properly licensed practitioner).

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Manufacturer
DeRoyal
200 DeBusk Lane
Powell, TN 37849 U.S.A.
888.938.7828
Part #1-670G
Revision C (5/03)
©2003 DeRoyal. All Rights Reserved.

Table with columns: Pulgadas, Centímetros, Tamaño, Longitud. Rows: AA, A, B, C, D with measurements.

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ESPANOL

DEUTSCH

PRECAUTIONS
These instructions apply to product number: 670
1. Snap cover closed. Trim excess string. Be sure enough string is left for increased swelling.
2. Pull strings to obtain desired compression.
3. Remove from finger. Wrap three times or more for secure hold.
4. These products are to be fitted initially by a healthcare professional who is familiar with the purpose for which they are used...

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Table with columns: Inches, Metric, Size, Length. Rows: AA, A, B, C, D with measurements.

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INTENDED USE
Read completely before applying.
IMPORTANT
For total finger application, measure from the patient crease to thumb tip. Choose closest total length.
CONSTRUCTED OF A WASHABLE, SMOOTH, OPEN-WEAVE COMPRESSION COVER AND A FOAM SUPPORT.
Determine where foam support is to be placed. If discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.

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