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### IMPORTANTE

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Small female
Average female, small male
Average to large male
D 3 3/8" - 3 3/4"
C 2 7/8" - 3 1/4"
B 2 1/4" - 2 3/4"
5.71 - 6.98 cm
7.28 - 8.26 cm
8.57 - 9.53 cm
Wrist Support
The following information should be read prior to use of this device.
IMPORTANT
The wrist support uses wire coils at the wrist joint axis to assist the wrist and digits into extension. The thumb outrigger assists the wrist joint axis to assist the wrist and digits into extension. This splint is recommended in injuries involving the radial nerve.
Splint size is determined by the width of the hand at the metacarpophalangeal joints.

Directions for Use
1. To fit the splint, loosen all strapping from the radial side. Slide hand into splint so the contoured palm support rests in the palmar arch and the distal bar rests along the volar surface of the proximal phalanx of the fingers.
2. When strapping the elastic wrist strap, note that the strap remains fastened around the ulnar coil. Cross the strap over the wrist. Wrap the strap around the wire extending from the radial coil. Adhere the loop material to the hook material. Once fastened, the strap will rest along the dorsum of the wrist.
3. On the wrist, attach the folded end of the DuraVal® to the radial side of the splint. Secure DuraVal® to the radial side of the splint. Secure the splint at the wrist by pulling the strap through the ulnar slit and adhering it to the adjacent hook material. Securing the splint will position the ulnar support close to the wrist. Professionals, care providers involved in the patient's care, and the patient, if unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.

Precautions
These products are to be fitted initially by a healthcare professional who is familiar with the purpose for which they are used. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare providers, care providers involved in the patient's care, and the patient. If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.

Care Instructions
Straps may be removed and laundered. Splint surface may be cleaned with a mild, non-abrasive detergent and water. Air dry prior to wearing. Single Patient Use

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©2003 DeRoyal. All Rights Reserved.