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#### IMPORTANTE

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**1. Determine the joint on the fingers where the finger separator will be placed (MP, DIP, or PIP). Note:** Open the hook and loop closure.

**DIRECTIONS FOR USE**  
Check the fingers for the desired position. Assess fingers for adequate circulation. If your facilities' protocol.

Severe or fixed contractures; excessive edema or swelling; excessive pain.  
Mild to moderate contractures of the fingers.

**INDICATIONS**  
This product is intended to help position contract or deviated fingers.

**INTENDED USE**  
Please read all instructions, warnings and precautions before use.

**IMPORTANT**

2. Carefully open the fingers and insert them into the loops of the finger separator, and position at the desired finger joint. **Note:** Do not put the thumb into a loop of the finger separator. **IMPORTANT:** Please read all instructions, warnings and precautions before use.

3. Once the orthosis is fitted, check the skin at the desired location for adequate circulation. If your facilities' protocol.

4. If using the finger separator with a hand orthosis, apply as in steps 1 – 2. Apply the care providers involved in the patient's care, and the patient. Consult your medical professional immediately if you experience swelling or prolonged pain while using this product. The Pucci Finger Separator should be removed routinely and skin assessment performed to check for any redness or irritation of the skin, excessive swelling, or excessive pain.

**PRECAUTIONS**  
These products are to be fitted initially by a healthcare professional who is familiar with the purpose for which they are used. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare professionals. Please read all instructions, warnings and precautions before use.

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**CARE INSTRUCTIONS**  
Machine wash in warm water and mild detergent. Rinse and air dry.  
Single Patient Use

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**Part #1-812G**  
Revision D  
Revised 5/03  
©2003 DeRoyal. All Rights Reserved.

**Manufacturer**  
DeRoyal  
200 DeBusk Lane  
Powell, TN 37849 U.S.A.  
888.938.7828

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**MediRoyal Nordic AB**