



DeRoyal Functional Ankle Brace (DFAB)

For Single Patient Use Only



IMPORTANT INFORMATION

The following information should be read prior to use of this device.

INTENDED USE

This functional ankle brace is intended to provide medial and lateral support to the ankle joint while allowing plantar and dorsiflexion.

INDICATIONS


Acute ankle sprains, chronic ankle sprains, and rehabilitation.

PRECAUTIONS

Consult your medical professional immediately if you experience sensation changes, unusual reactions, swelling or prolonged pain while using this product.

DIRECTIONS FOR USE

1. Position the brace so the sticker reading *outside of ankle* is on the lateral (outside) of the injured ankle. (A)
2. Lift the insole (or orthotic) of your athletic shoe and place the brace inside the shoe. **Note:** The footplate padding can be removed for a better fit. **Note:** If insole is not removable, the brace may be worn above the insole keeping the footplate padding intact. (B)
3. Place your foot inside the shoe. The hinge should be centered at the mid-point of the ankle bone (malleoli).

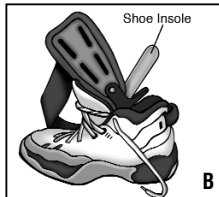

4. Secure the lower strap followed by the upper strap. The straps are snug, but comfortable. (C)
5. Secure the shoe strings on your athletic shoes. **Note:** The brace should be worn in an athletic shoe and with an athletic sock.



Caution: Federal U.S.A. law restricts this device to sale or use by or on the order of a physician (or properly licensed practitioner).

SIZING CHART

Shoe Size	Men's	Women's	
			X-Small
		5 – 6 1/2	Small
5 – 8 1/2	7 – 9 1/2		Medium
9 – 12	10 – 13		Large
12 1/2 – 16	13 1/2 – 16		



Manufacturer
DeRoyal
200 DeBusk Lane, Powell, TN 37849 U.S.A.
888.938.7828

Authorized Representative
DeRoyal Europe, Ltd.
Virginia Road, Kells, County Meath Ireland
353.46.9280100

Part #0-857
Revised 5/03
©2003 DeRoyal. All Rights Reserved.



DeRoyal Functional Ankle Brace (DFAB)

For Single Patient Use Only



IMPORTANT INFORMATION

The following information should be read prior to use of this device.

INTENDED USE

This functional ankle brace is intended to provide medial and lateral support to the ankle joint while allowing plantar and dorsiflexion.

INDICATIONS


Acute ankle sprains, chronic ankle sprains, and rehabilitation.

PRECAUTIONS

Consult your medical professional immediately if you experience sensation changes, unusual reactions, swelling or prolonged pain while using this product.

DIRECTIONS FOR USE

1. Position the brace so the sticker reading *outside of ankle* is on the lateral (outside) of the injured ankle. (A)
2. Lift the insole (or orthotic) of your athletic shoe and place the brace inside the shoe. **Note:** The footplate padding can be removed for a better fit. **Note:** If insole is not removable, the brace may be worn above the insole keeping the footplate padding intact. (B)
3. Place your foot inside the shoe. The hinge should be centered at the mid-point of the ankle bone (malleoli).

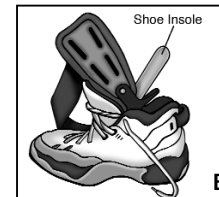

4. Secure the lower strap followed by the upper strap. The straps are snug, but comfortable. (C)
5. Secure the shoe strings on your athletic shoes. **Note:** The brace should be worn in an athletic shoe and with an athletic sock.



Caution: Federal U.S.A. law restricts this device to sale or use by or on the order of a physician (or properly licensed practitioner).

SIZING CHART

Shoe Size	Men's	Women's	
			X-Small
		5 – 6 1/2	Small
5 – 8 1/2	7 – 9 1/2		Medium
9 – 12	10 – 13		Large
12 1/2 – 16	13 1/2 – 16		



Manufacturer
DeRoyal
200 DeBusk Lane, Powell, TN 37849 U.S.A.
888.938.7828

Authorized Representative
DeRoyal Europe, Ltd.
Virginia Road, Kells, County Meath Ireland
353.46.9280100

Part #0-857
Revised 5/03
©2003 DeRoyal. All Rights Reserved.