## DeRoyal Air Ankle Stirrup

Single Patient Use Only

Rx Only • Latex Safe

### SIZING

	Height	Product Number
Standard	10"	AB2360-00
Small	8.5"	AB2361-00
Pediatric	6"	AB2362-00

### IMPORTANT INFORMATION

Please read all instructions, warnings and precautions before use. Correct application is essential to proper functioning of product.

### INDICATIONS

Acute ankle sprains and strains, chronic instabilities and prophylactic use.

### PRECAUTIONS

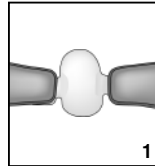
- This product is to be fitted by a physician (or properly licensed practitioner) who is familiar with the purpose for which they are responsible. The physician or practitioner is responsible for providing wearing instructions and precautions to other healthcare practitioners or healthcare providers involved in the patient's care and the patient.
- Consult your physician (or properly licensed practitioner) immediately if you experience sensation changes, unusual reactions, swelling

or prolonged pain while using this product.

- Follow the instructions of your physician (or properly licensed practitioner) for length and duration of use.

### DIRECTIONS FOR USE


- Spread the shells and place the patient's heel on the heel strap (located between the shells). **NOTE:** Before application place an absorbent sock or ankle wrap on patient's ankle.


- Place the shells against the sides of the ankle.

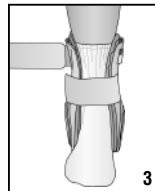- Compress the shells around the patient's ankle, while securing straps from bottom to top.

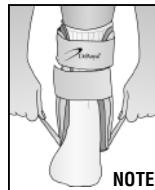**NOTE:** To adjust heel strap for a snug fit, raise or lower the heel straps located on the outer side of the shells.


- Firmly insert clear end of inflation bulb into valve and inflate bladder per physician's order by squeezing the bulb until desired compression is achieved. **NOTE:** The pediatric ankle brace air cells are pre-inflated and do not require adjustment.

- Check for adequate circulation. Readjust air amount and straps as needed for a secure and comfortable fit.

- Insert patient's foot into lace-up shoe (unless otherwise instructed).

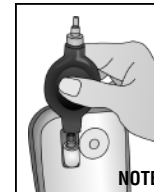**NOTE:** To deflate air bladder, firmly insert black end of inflation bulb into valve and deflate bladder per physician's order by squeezing the bulb.

### WASHING INSTRUCTIONS

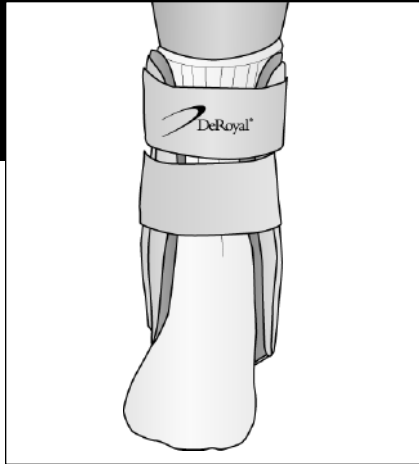
Wipe down ankle brace with a damp cloth and air dry.

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**MediRoyal Nordic AB**