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FOTLEDSORTOS, SNÖRAD MODELL, DSAB

CE

1. Place foot into brace and lace from the bottom up securely, but comfortably. For proper fitting, apply the brace with comfortably positioned at a 90 degree angle. If desired, support stays can be removed from the brace. **NOTE:** DO NOT tighten the laces so tightly that circulation is restricted. If circulation is restricted, loosen the strap.
2. Apply tension to the outside strap (Strap #1 in the illustration) by pulling the strap in an upward fashion. While tension is being applied, roll ankle to the inside toward the opposite leg and attach the strap to the outside of the brace. **NOTE:** This strap should be fed through the guide on the outside of the brace to ensure proper positioning (see illustration). **CAUTION:** DO NOT apply too much tension to the strap to cause restriction of circulation. If circulation is restricted, loosen the strap.
3. Apply tension to the figure eight strap (Strap #3 in illustration) by pulling the strap across the top of the foot toward the inside of the ankle, around the bottom of the foot to the outside of the ankle and back across the top of the foot to attach on the inside of the brace. (This should form a "figure eight"). **CAUTION:** Strap #3 should not be so tight that it causes cramping or pain in the arch of the foot. If cramping occurs, the strap is too tight and should be loosened. **NOTE:** Straps #1 and #3 may be cut off if they are too long. Apply the brace and ensure proper fit, then cut straps the desired length.
4. Wrap the elasticized holding strap (Strap #4 in illustration) around the leg and secure as shown. **CAUTION:** DO NOT apply too much tension to the strap to restrict circulation. If circulation is restricted, loosen strap.

Single Patient Use

1. Place foot into brace and lace from the bottom up securely, but comfortably. For proper fitting, apply the brace with comfortably positioned at a 90 degree angle. If desired, support stays can be removed from the brace. **NOTE:** DO NOT tighten the laces so tightly that circulation is restricted. If circulation is restricted, loosen the strap.
2. Apply tension to the outside strap (Strap #1 in the illustration) by pulling the strap in an upward fashion. While tension is being applied, roll ankle to the inside toward the opposite leg and attach the strap to the outside of the brace. **NOTE:** This strap should be fed through the guide on the outside of the brace to ensure proper positioning (see illustration). **CAUTION:** DO NOT apply too much tension to the strap to restrict circulation. If circulation is restricted, loosen strap.
3. Apply tension to the figure eight strap (Strap #3 in illustration) by pulling the strap across the top of the foot toward the inside of the ankle, around the bottom of the foot to the outside of the ankle and back across the top of the foot to attach on the inside of the brace. (This should form a "figure eight"). **CAUTION:** Strap #3 should not be so tight that it causes cramping or pain in the arch of the foot. If cramping occurs, the strap is too tight and should be loosened. **NOTE:** Straps #1 and #3 may be cut off if they are too long. Apply the brace and ensure proper fit, then cut straps the desired length.
4. Wrap the elasticized holding strap (Strap #4 in illustration) around the leg and secure as shown. **CAUTION:** DO NOT apply too much tension to the strap to restrict circulation. If circulation is restricted, loosen strap.

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