mediroyal 

# Hand Mitten

English  
Slovenščina

GB

SI

© MediRoyal Nordic AB  
Box 7052 · SE-192 07 Sollentuna · Sweden  
+46 8 506 766 00 · [www.mediroyal.se](http://www.mediroyal.se)

GB

## English – Instruction

### Mediroyal Hand Mitten for upper limb control

#### General information

Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, registered nurse, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the textile label.

#### Indications

The product has been designed to control the upper limbs in order for practitioner to aid in controlling hand and finger or limb movements to prevent patients from disturbing medical treatment.

#### Contraindications

Do not use on a patient who is or may become highly aggressive or agitated. Never use on a patient if an I.V. or wound site could be compromised by the device.

#### Pre Application

The hand mittens are universal in size and made of a breathable and comfortable material that is not flame retardant. The physician or practitioner is responsible for providing wearing instructions and precautions to other healthcare practitioners or healthcare providers involved in the patient's care as well as the patient. Consult your physician (or properly licensed practitioner) immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this product. Follow the instructions of your physician (or properly licensed practitioner) for length and duration of use. Be sure to follow your country's legislation for restraining patients and the facility's policies and guidelines for frequency of patient monitoring.

#### Application

The mitten is made of a breathable and comfortable material that is not fire resistant. Place the patient's hand in the mitten. Ensure proper finger placement in the assigned finger separators. Secure the wrist closure using hook and loop closure. Do not over tighten. Two fingers should fit between the device and the patient's wrist to ensure circulation is not compromised. If the hand mitten needs to be used as a more restrictive device, the bed attachment strap maybe used. To apply the bed attachment strap, feed the end of the strap through the loop on the wrist of the mitten. Feed the end of the strap through the sea-wave buckle and adjust the strap, so the buckle is positioned near the loop. Secure the strap to a part of the bed that moves with the patient. Make sure that the end of the strap is out of the patient's reach.

#### Removal

Open the hook and loop closure around the wrist and remove the mitten.

#### Caring instructions, warnings and precautions

Improper application or use of any restraint may result in serious injury or death. Secure straps to a portion of the bed that moves with the patient. Do not secure straps to bed side rails. Keep straps out of patient's reach. Patient should not be able to remove device once fitted. Make sure the bed side rails are up while product is in use. Check the patient regularly to ensure circulation is not compromised or that there are no pressure problems on the hands.

We recommend that the mittens should be washed once a week. Follow the washing instructions on the white label, use liquid detergent, close the straps and always use a laundry bag. Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on label on the product.

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