GB English – instructions

Knee Supports – Patella Stab

General information · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the backside of the white textile label.

Indications · The product has been designed to support the patella either from the lateral or medial side.

Contraindications · If you haven’t received a diagnosis from a doctor should consult one before use of the brace.

Pre Application · To select the right size, take the circumference measurement around the knee at the height of the knee cap. Use the measurement chart to determine the most suitable size. Open all Velcro straps of the brace before application.

Application · It’s important that the pad is placed correctly on either the outside of the patella or the inside. When applying the brace, make sure that the pad is close to the rim of the patella, almost touching it. If the brace has an open design, position the pad, then close the brace. If the brace is pulled up on the leg, make sure that you can feel the pad touching the patella. Adjust the support with the straps attached to the pad. Try walking, the support should be felt where the pad is but it shouldn’t hurt or disturb normal walking.

Removal · Open the Velcro straps and remove the brace. Close all straps when the brace is not in use.

Material content · Check the backside of the white label for material content.

Caring instructions and precautions · We recommend that the brace should be washed once a week. Follow the washing instructions on the white label on the product, close all straps and use a laundry bag.

Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on the backside of the label.

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CE MediRoyal Nordic AB