GB English – instructions

MediRoyal Hallux Valgus Strap

General information · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the white textile label.

Indications · Hallux Valgus

Pre Application · The product is made universal in size but in left and right

Application · Apply the toe piece over the big toe. The short part with the hook piece should be on the upper side. Let the longer part run under the foot and around the heel. If you are using it when you are very active, you can run a second loop around the ankle before you attach the strap onto the hook piece.

The tension needed to straighten the toe is very individual. The best is to start with low tension and gradually increase it.

Function · The strap can be used both bare-foot as well as with shoes on, as long as there is space enough in the shoe. The function of the strap is to position the toe in a more functional position which can decrease pain. When use the strap in activity, the abductor and adductor muscles of the big toe will gain strength which can assist in slowing down the hallux valgus process. It's important to combine the strap with foot exercises to reach an optimal effect. Contact a physical therapist or foot therapist for more advice.

Removal · Open the Velcro strap and remove the strap. Always then check for pressure marks.

Caring instructions and precautions · We recommend that the strap should be washed once a week. Follow the washing instructions on the label and always use the laundry bag. Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on the label.

More information can be found on www.mediroyal.se

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© Copyright MediRoyal Nordic AB · Sweden · 2015 · www.mediroyal.se
MediRoyal Nordic AB Box 7052, SE-192 07 Sollentuna Sweden +46 8 506 766 00 www.mediroyal.se