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www.mediroyal.se

## GB The ErixTwo Shoulder Brace - English

The ErixTwo shoulder brace by MediRoyal is a shoulder brace that combines mechanical stability with proprioceptive feed-back. The brace features elastic loop straps in combination with a brace with loop surface and silicone stripes on the inside. The elastic loop straps on the outside can provide individual support and stability as well as transfer forces from the outside of the brace to the skin surface. This leads to a higher bio-feedback and proprioception. The ErixTwo Shoulder Brace is made of NeoTex that insulates body heat. For further modular stability, see the accessories to The ErixOne Shoulder Brace. They are interchangeable and can be used for The ErixTwo Shoulder Brace as well.

### Application

Start with selecting the right size with the help of the measurement chart. You will get the best effect if the brace is used directly onto the bare skin. The inside has silicone straps that will grip the skin for a proprioceptive effect. If you find this to be uncomfortable a t-shirt can be worn under.

- Open the buckle on the front side of the brace. Open all hook and loop closures on the arm piece then slide the brace up the arm.
- Close the hook and loop straps on the arm piece. Make sure that the tongue covers the skin when the hook straps are attached to avoid discomfort.
- Grab the buckle on the back side.
- Pre-stretch the back side of the brace by pulling the strap. This step is important for the function of the brace.
- Attach it under the arm-pit maintaining the pre-stretch in the strap. The material will grip the skin and will help to hold the brace in place.
- Secure the pre-stretched strap by pinching it with your upper arm against the body.
- Attach the buckle and open the hook on the strap.
- Adjust the strap and close the hook.
- When the brace has been correctly applied you should now feel a slight pull from the back side.
- Now adjust the straps on the upper side of the brace. Start with the strap in the back. You might need assistance with that since the best effect is reached when the brace has been secured and then adjusted. Pull the strap until it feels comfortable and secure the hook.
- Then adjust the front strap and secure the hook.
- The center strap is the last one to be adjusted. Slightly lift the arm up if possible and then add some tension to the strap before securing it.
- The brace provides stability and compression to the shoulder joint up to 90 degrees abduction. Further stability and range of motion restrictions can be reached by using the modular system from The ErixOne Shoulder Brace like chest belt, straps and pads. See the separate information for The ErixOne Shoulder Brace by MediRoyal.

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