

# MCP Extension Assist

**Material**

NRX<sup>®</sup> 30 mm and 50 mm + 2 hook tabs



Cut a 50 mm NRX<sup>®</sup> strap to fit the circumference of the wrist plus 2-3 cm. Then cut a 30 mm NRX<sup>®</sup> strap, approx 5-6 cm long. Round one of the ends and cut a small hole 3-4 mm from the edge. The hole should fit over the finger without restricting the blood flow.



Attach the 50 mm NRX<sup>®</sup> strap over the wrist and close with the hook tab.



Attach the strap onto the Manex Radial platform. Make sure that the MCP joints are neutrally aligned and not over extended.



Attach the hook part to the wrist strap. Adjust the length of the finger extension strap if needed. Test the functionality of the patient. If needed adjust the tension.

**Caring instructions**

The NRX<sup>®</sup> strap can be washed in 40 degrees machine wash with a liquid detergent. Always use a laundry bag. Remove the hook parts before washing and let air dry.